



RF ENCLOSURE SHOP DRAWING

Page	Page Description
0	Drawing Sheet Index
1	Top view
2	Magnet Position
3	Ground opening
4	Lightning Panels- Anemostat
5	View -A Structural
6	View -A Interior Finishing
7	View -B Structural
8	View -B Interior Finishing
9	View -C Structural
10	View -C Interior Finishing
11	Suspended Ceiling
12	CableTray
13	Wall Panels Top View
14	Wall Panels Fronte View-A
15	Wall Panels Left View-B
16	Wall Panels Behind View-C
17	Wall Panels Right View-D
18	Wall Panels Details
19	Ceiling Panels Details
20	Lower floor ground 18mm MDF
21	Upper floor ground 18mm MDF
22	Magnet Shielding
23	Magnet Shielding-A
24	RF Enclosure Details, Wall Panel
25	<i>RF Enclosure Details, (Door)</i>
26	<i>RF Enclosure Details, (Window)</i>
27	System filter BOX
28	Room lighting drawing


Project No: 2020xxx	Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo , Egypt	Drawing Sheet Index	Page	 CONTRACTING & SERVICES
Rev No: 2	Date: 02.11.2022	Dwg By: -		0	
Scale 1/50	Size A3	All dimensions are millimeters.	Model Type Ingenia Ambition S		

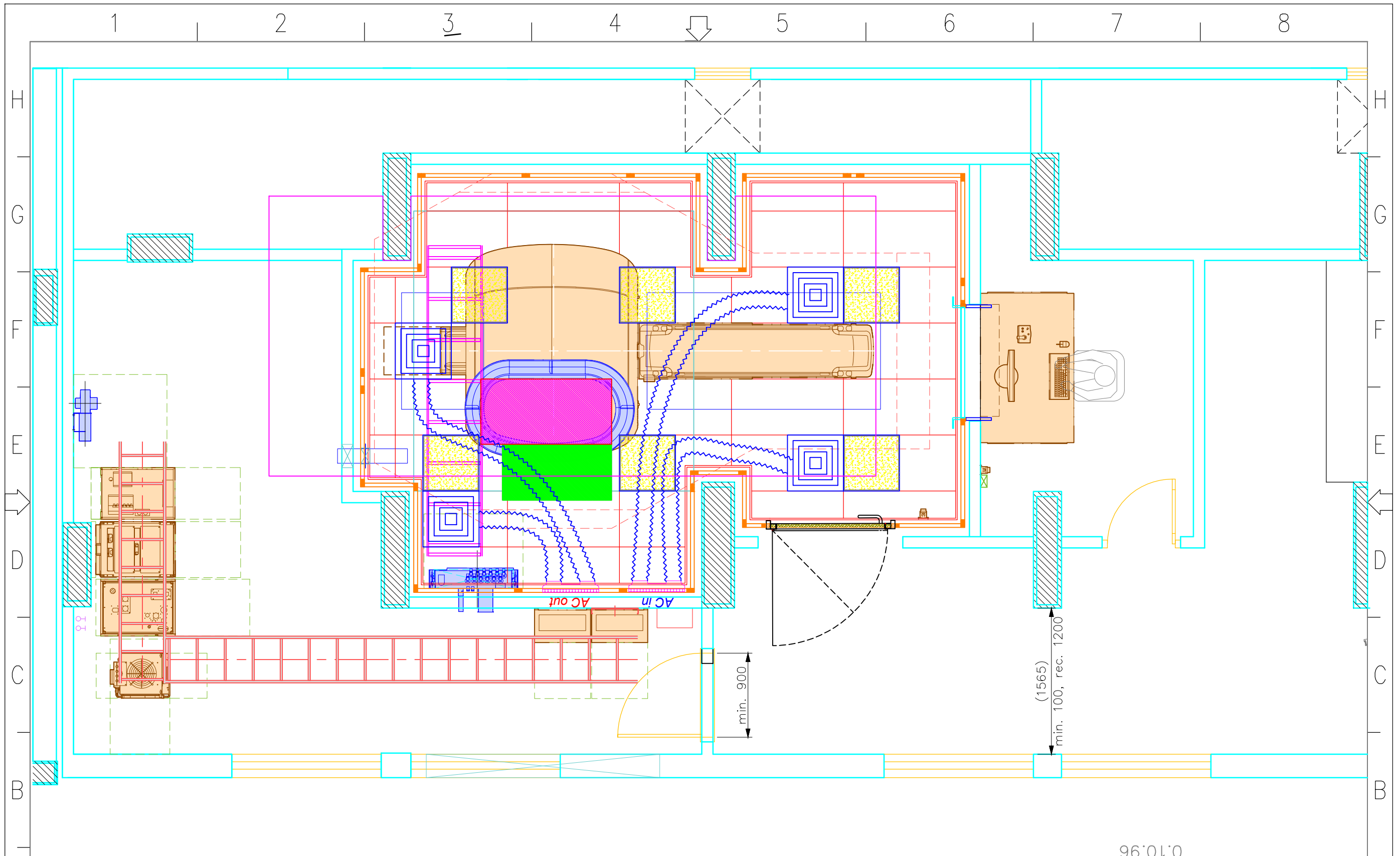


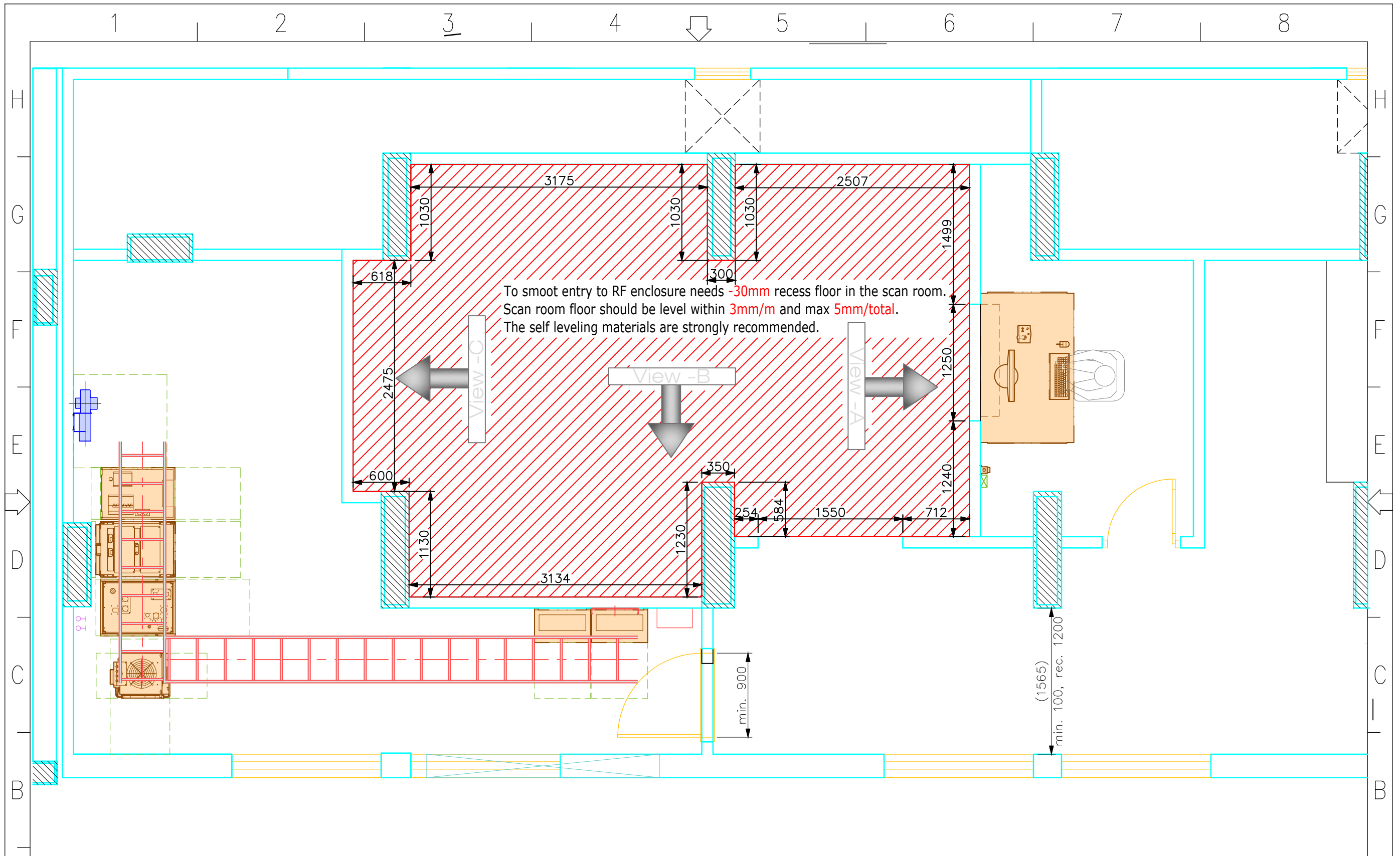
96'01"0


A	Project No: 2020xxx	Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo , Egypt	TOP VIEW	Page	 CONTRACTING & SERVICES	A
	Rev No: 2				1		
	Date: 02.11.2022	Dwg By: -	Model Type Ingenia Ambition S				
	Scale 1/50	Size A3	All dimensions are millimeters.				




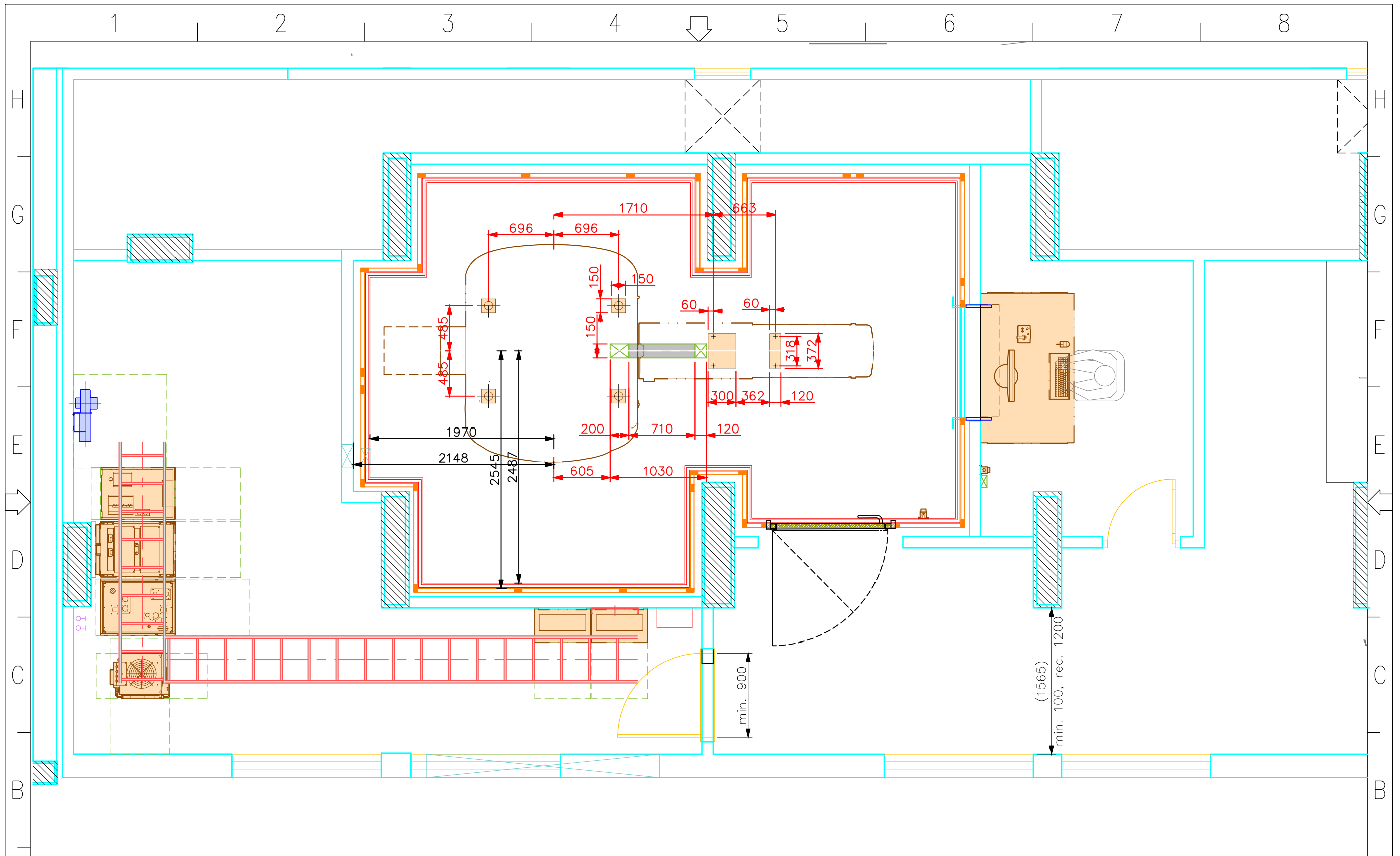


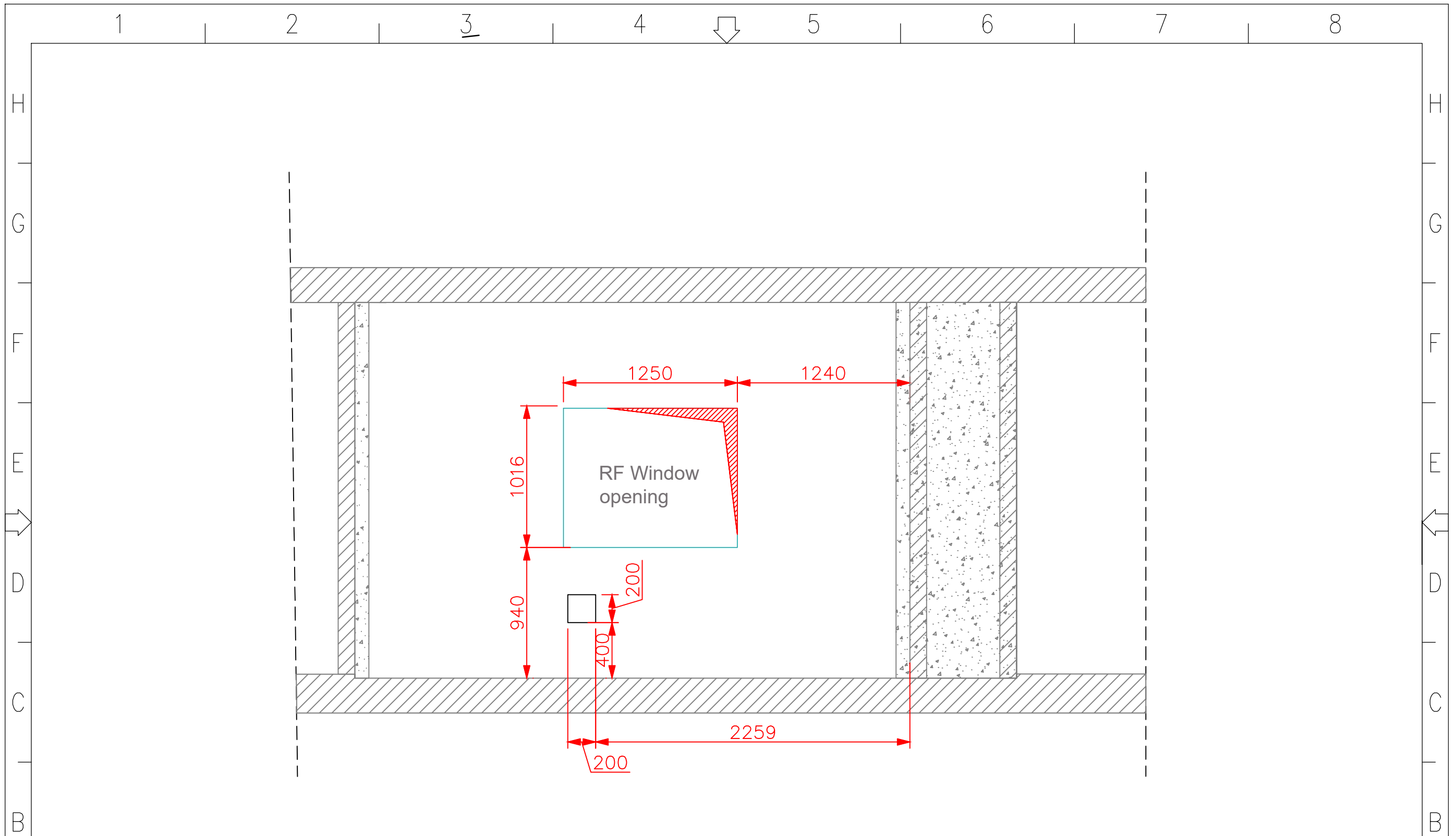
Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	<h1>Magnet Position</h1>	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		2	
Date: 02.11.2022	Scale 1/50		Size A3		All dimensions are millimeters.	




Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	Ground opening	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		3	
Date: 02.11.2022	Scale 1/50					

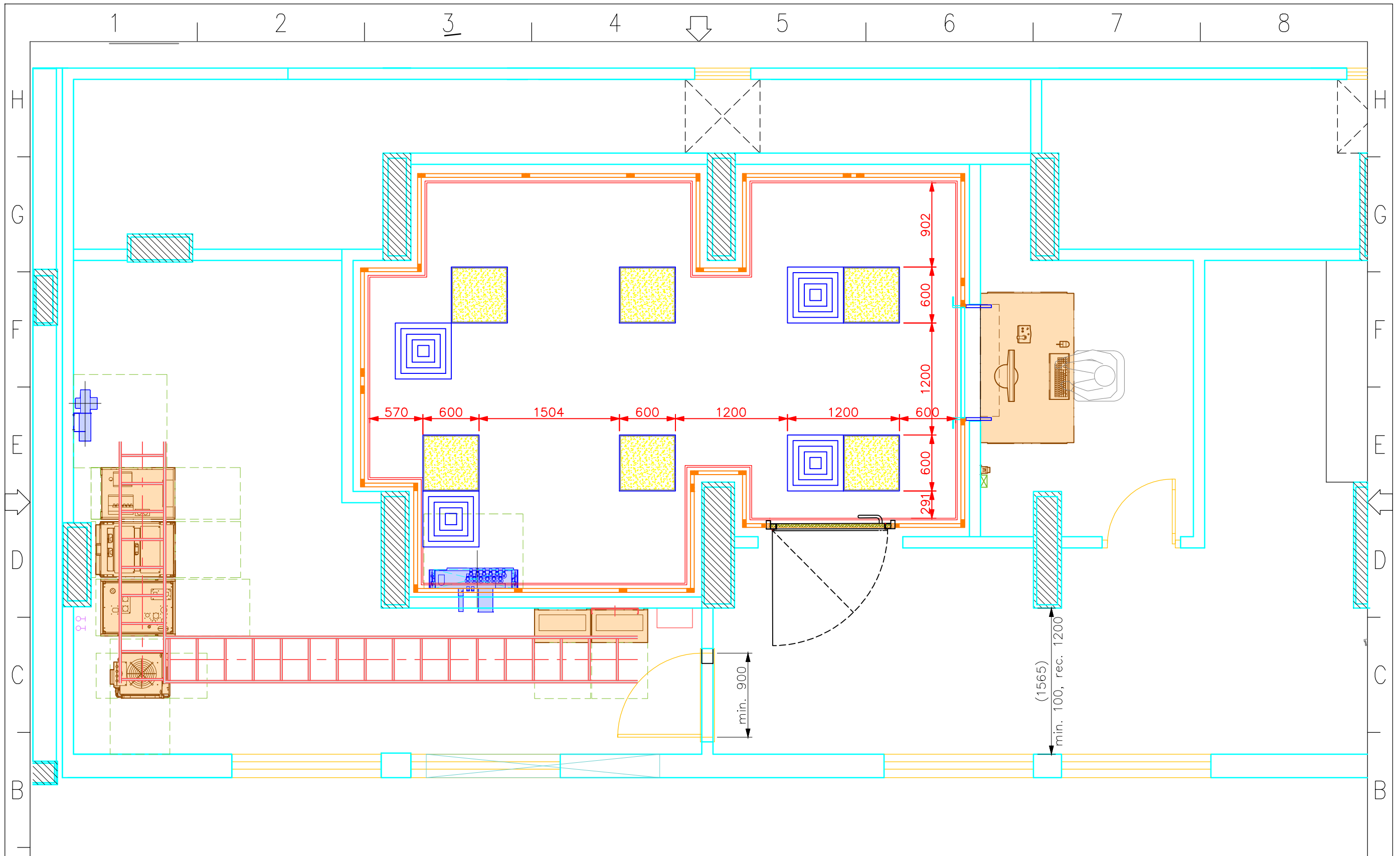


Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	Lightning Panels- Anemostat	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		4	
Date: 02.11.2022	All dimensions are millimeters.					
Scale 1/50	Size A3					




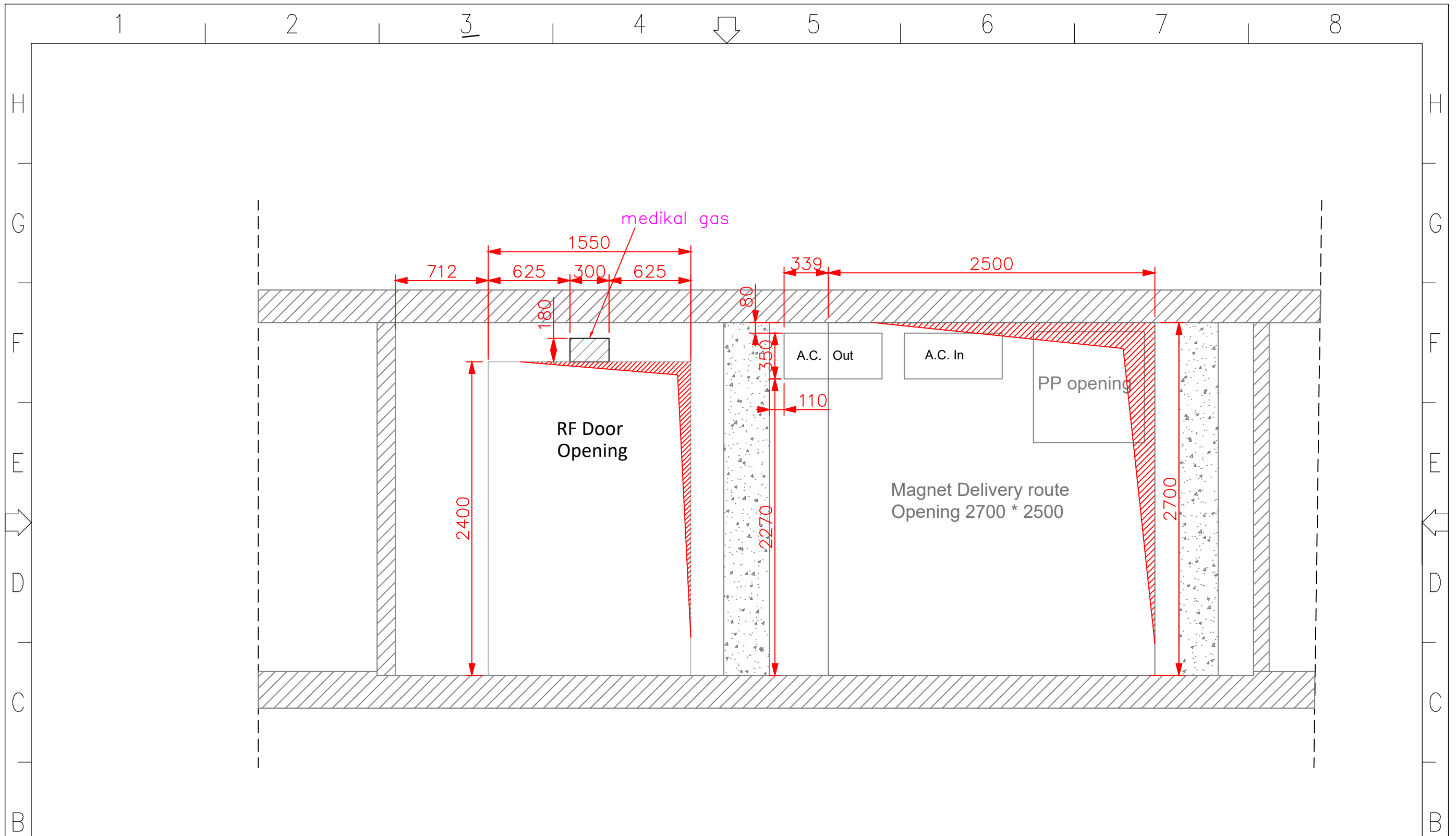
Note: All the scales are millimeters and from inside of the scan room.

Project No: 2020xxx		Dwg No: -		Customer/Project Name Cleopatra Hospital Cairo , Egypt		Page		 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -		Model Type Ingenia Ambition S		5		
Date: 02.11.2022		All dimensions are millimeters.		View -A Structural				
Scale 1/50	Size A3							

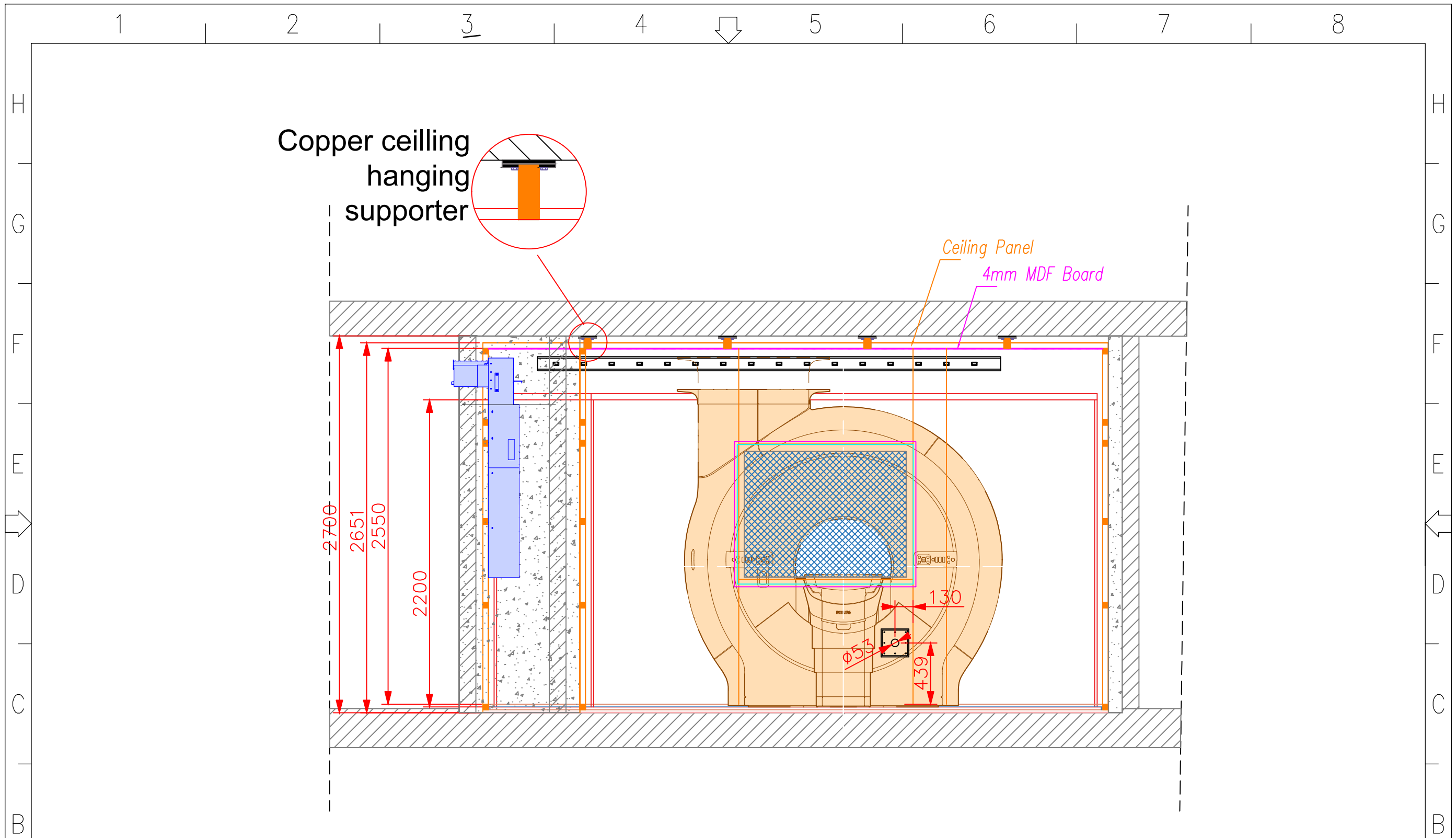


Note: RF viewed from the outside!


Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	View -A Interior Finishing	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		6	
Scale 1/50	Size A3	All dimensions are millimeters.				




Note: All the scales are millimeters and from inside of the scan room.


Project No: 2020xxx		Dwg No: -		Customer/Project Name Cleopatra Hospital Cairo, Egypt		Page		 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -		Model Type Ingenia Ambition S		7		
Scale 1/50	Size A3	All dimensions are millimeters.						

View -B Structural

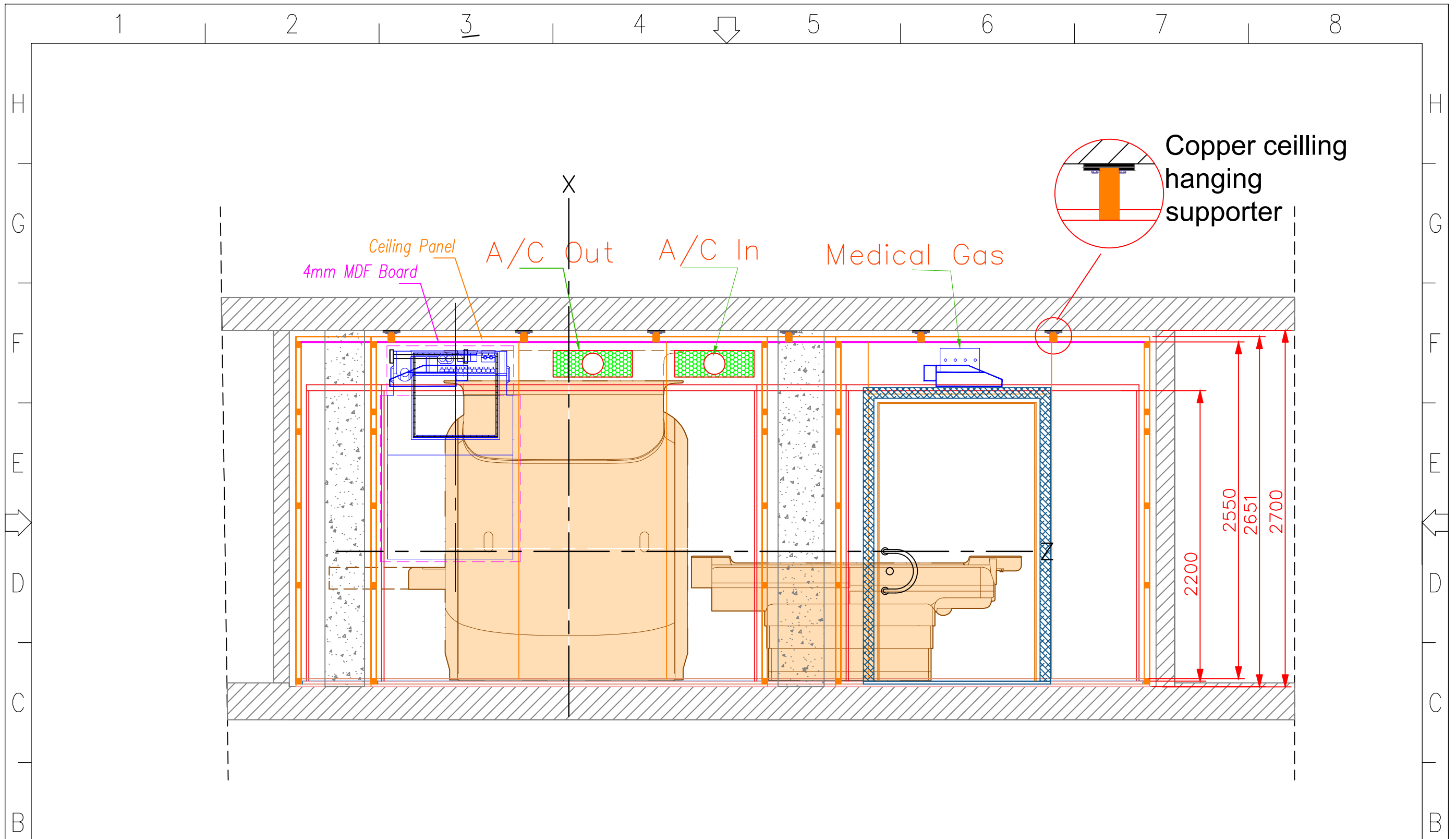


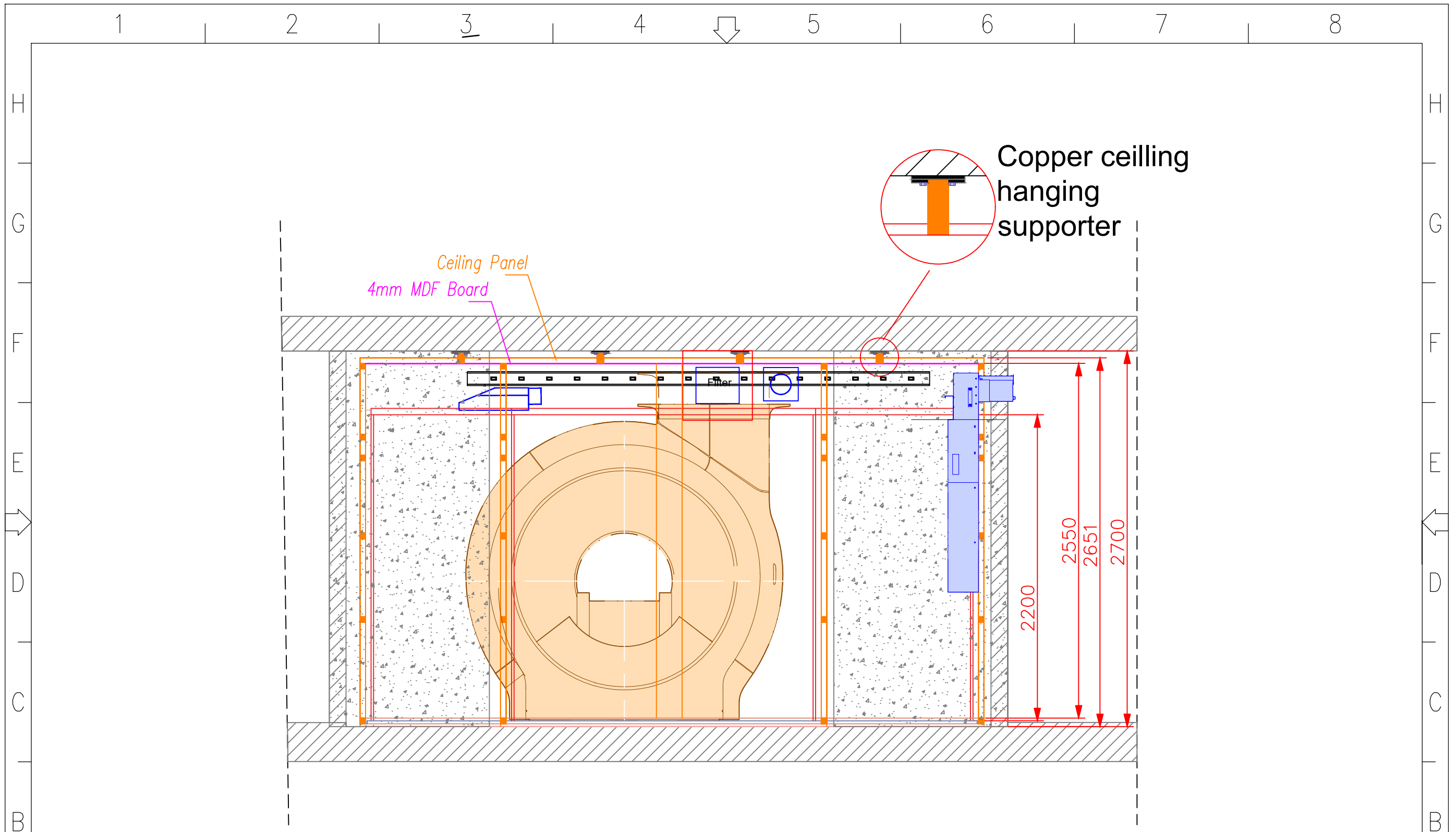
Note: RF viewed from the outside!

Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo , Egypt	View -B Interior Finishing	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		8	
Date: 02.11.2022	All dimensions are millimeters.					
Scale 1/50	Size A3					

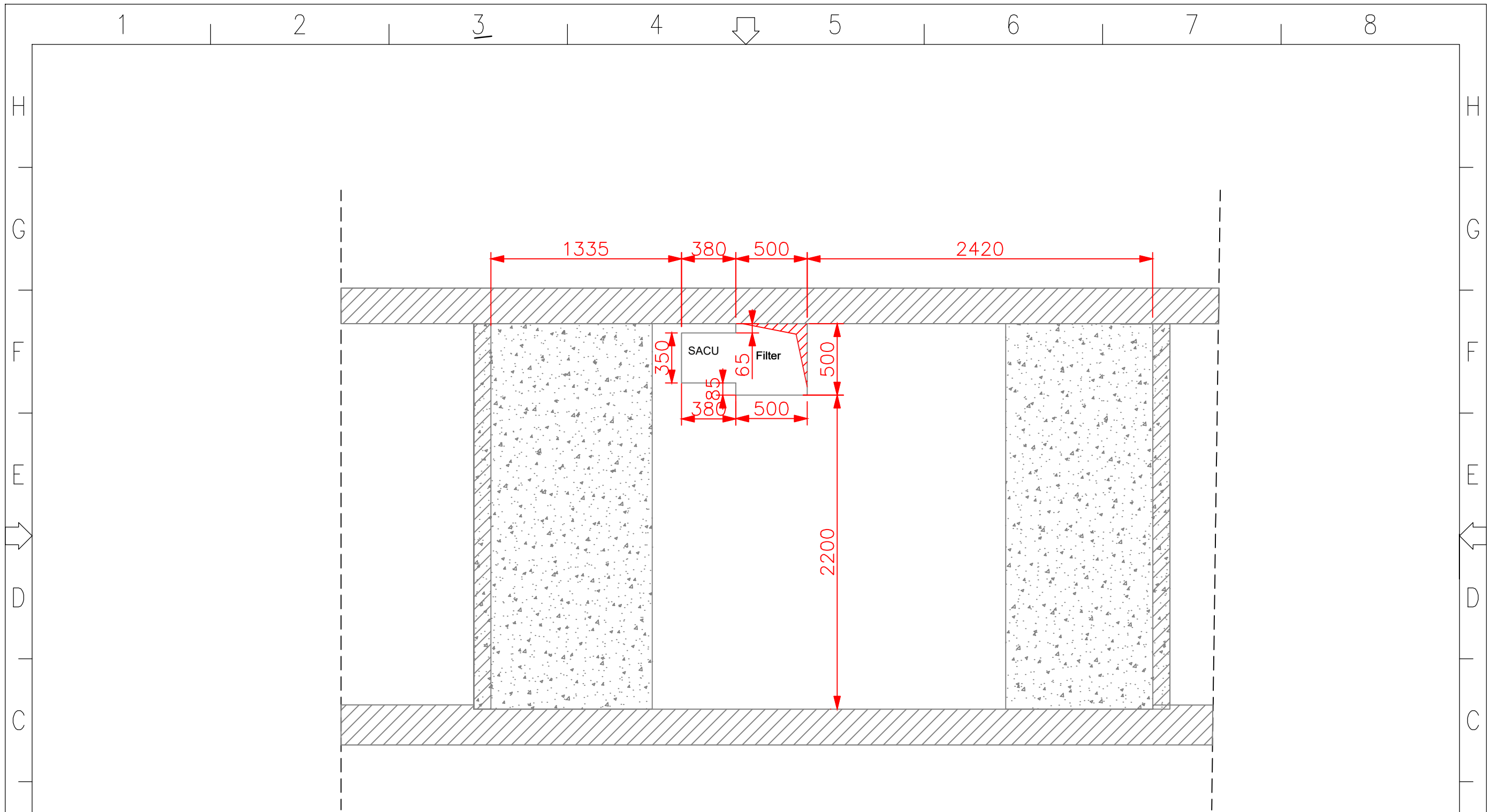


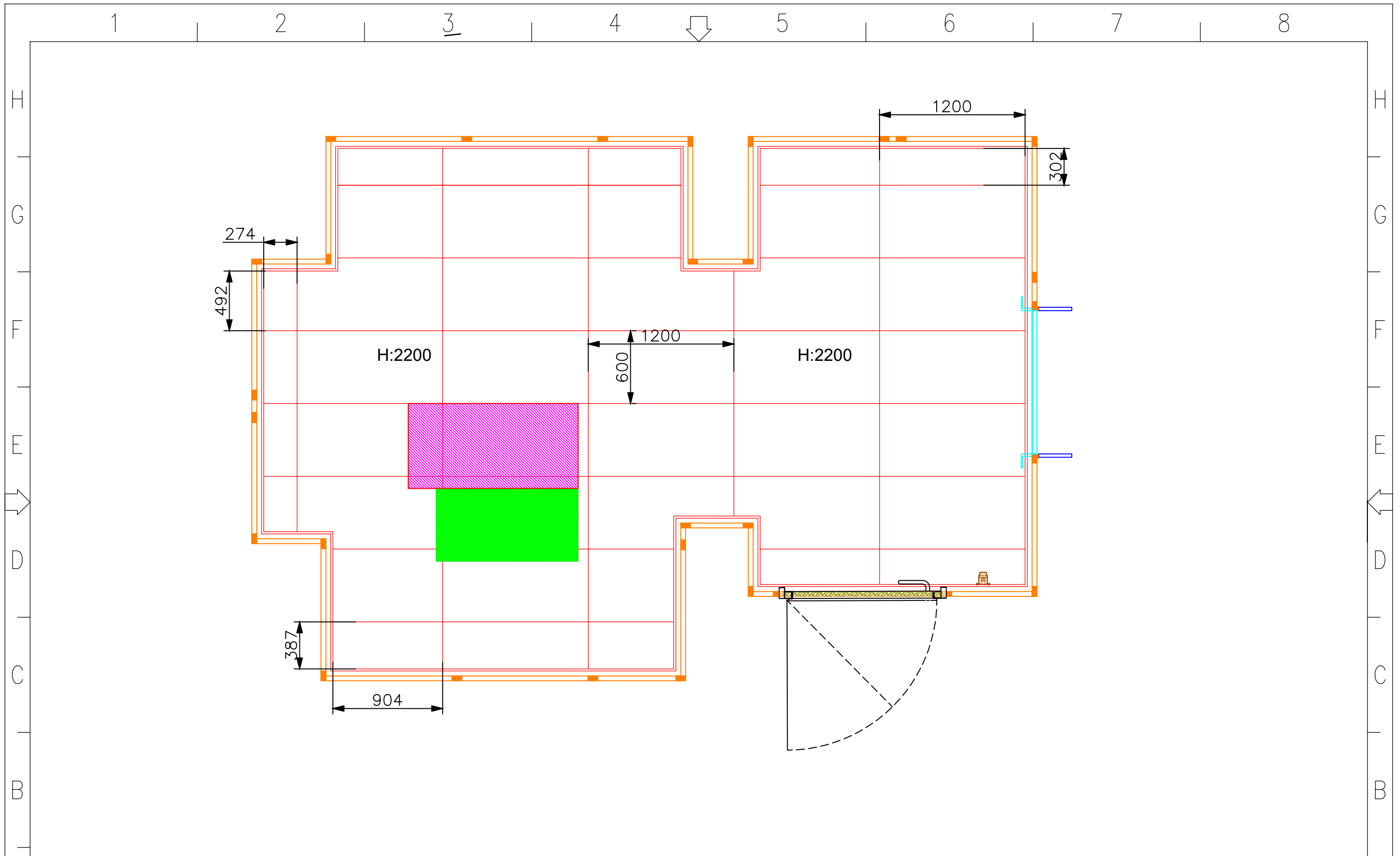
Note: All the scales are millimeters and from inside of the scan room.


Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo , Egypt	View -C Structural	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		9	
Scale 1/50	Size A3	All dimensions are millimeters.				




Note: RF viewed from the outside!

Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	View -C Interior Finishing	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		10	
Scale 1/50	Size A3	All dimensions are millimeters.				



Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	SUSPENDED CEILING	Page	 ceren CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		11	
Scale 1/50	Size A3	All dimensions are millimeters.				

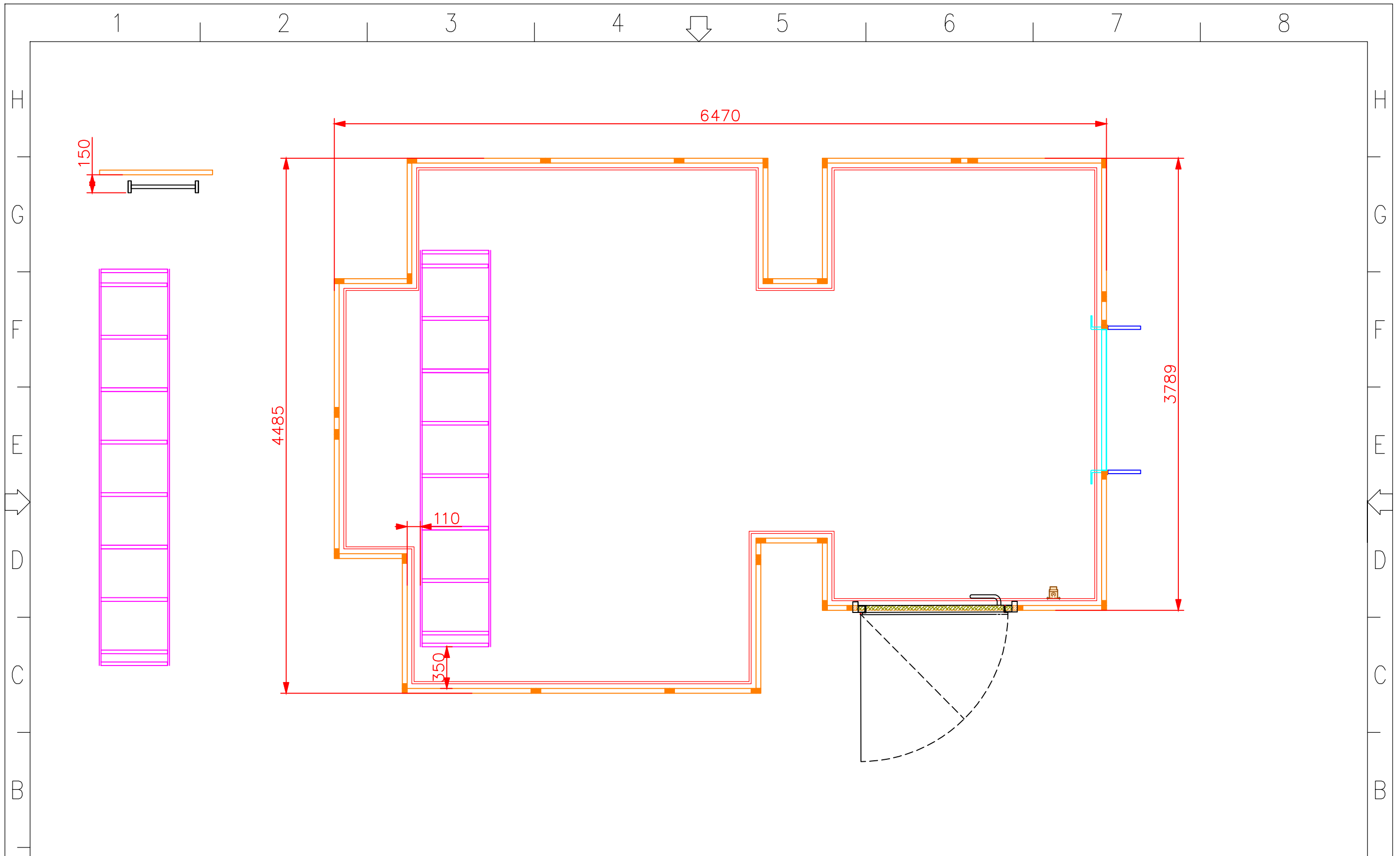

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Dwg No: -

Dwg By: -

All dimensions are millimeters.


Customer/Project Name
Cleopatra Hospital
Cairo , Egypt

Model Type
Ingenia Ambition S

CABLE TRAY

Page
12



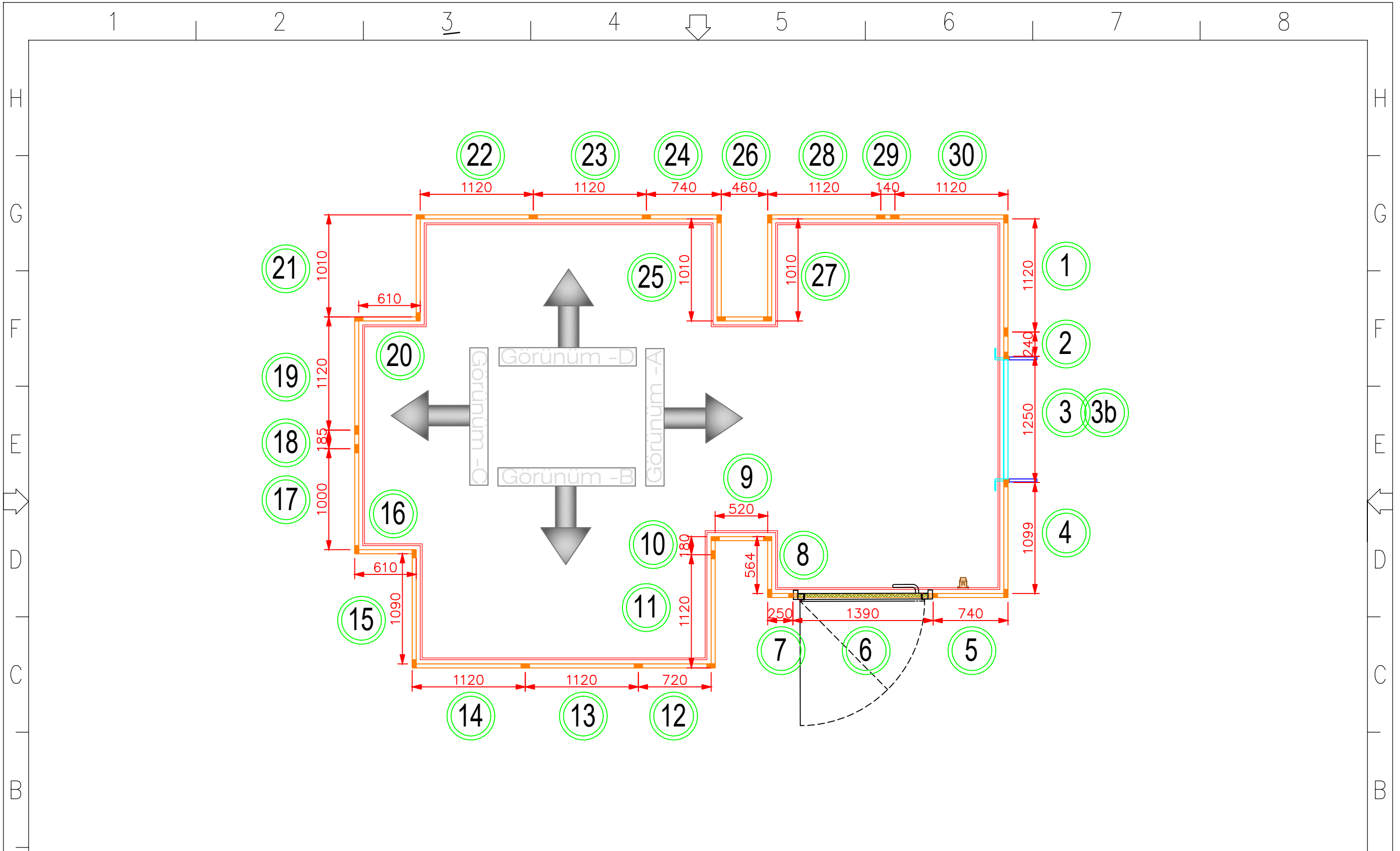

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Dwg No: -

Dwg By: -

All dimensions are millimeters.

Customer/Project Name
Cleopatra Hospital
Cairo, Egypt


Model Type
Ingenia Ambition S


WALL PANELS TOP VIEW




A

A

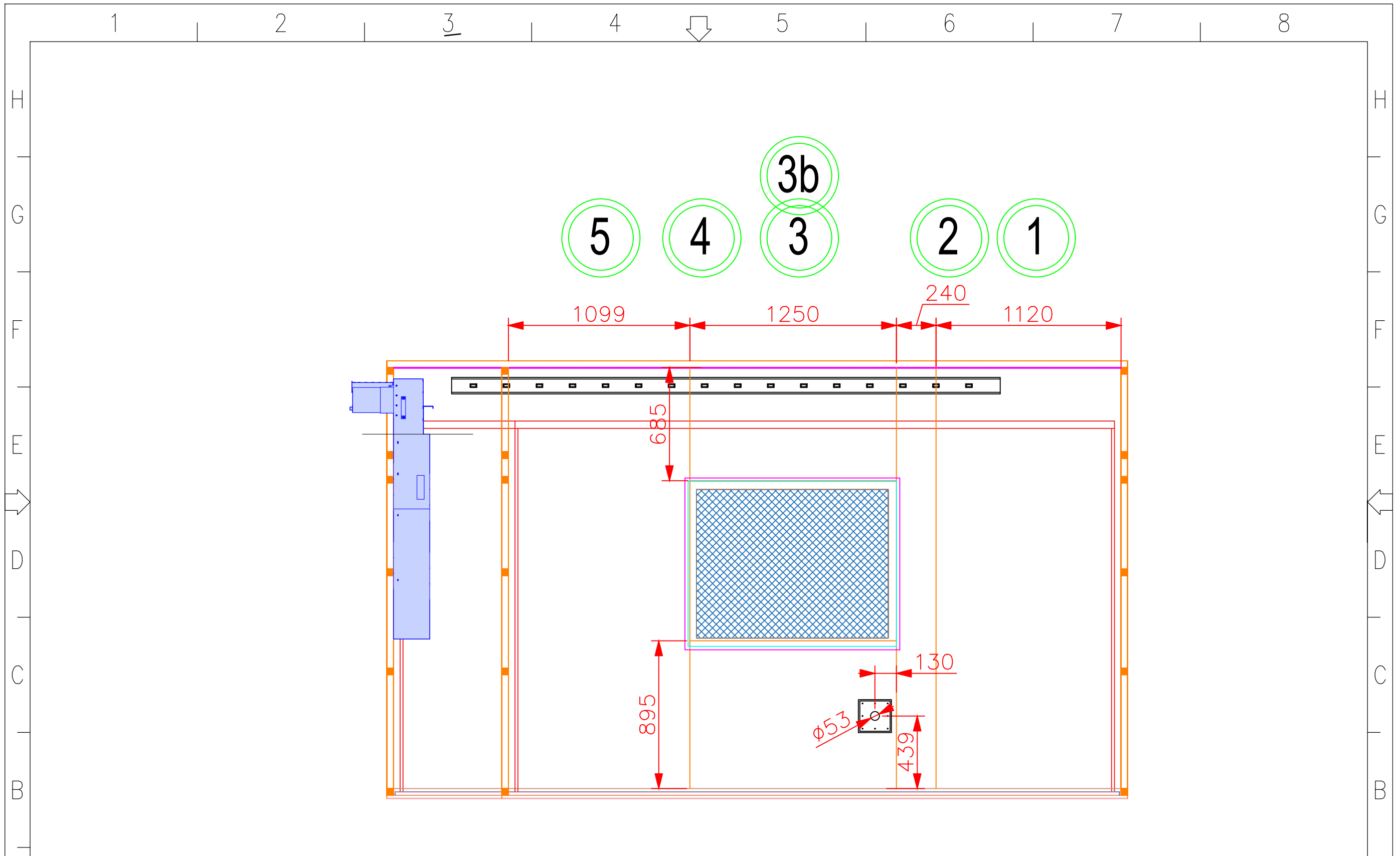

Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	WALL PANELS FRONTE VIEW-A	Page	 CONTRACTING & SERVICES
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		14	
Scale 1/50	Size A3	All dimensions are millimeters.				

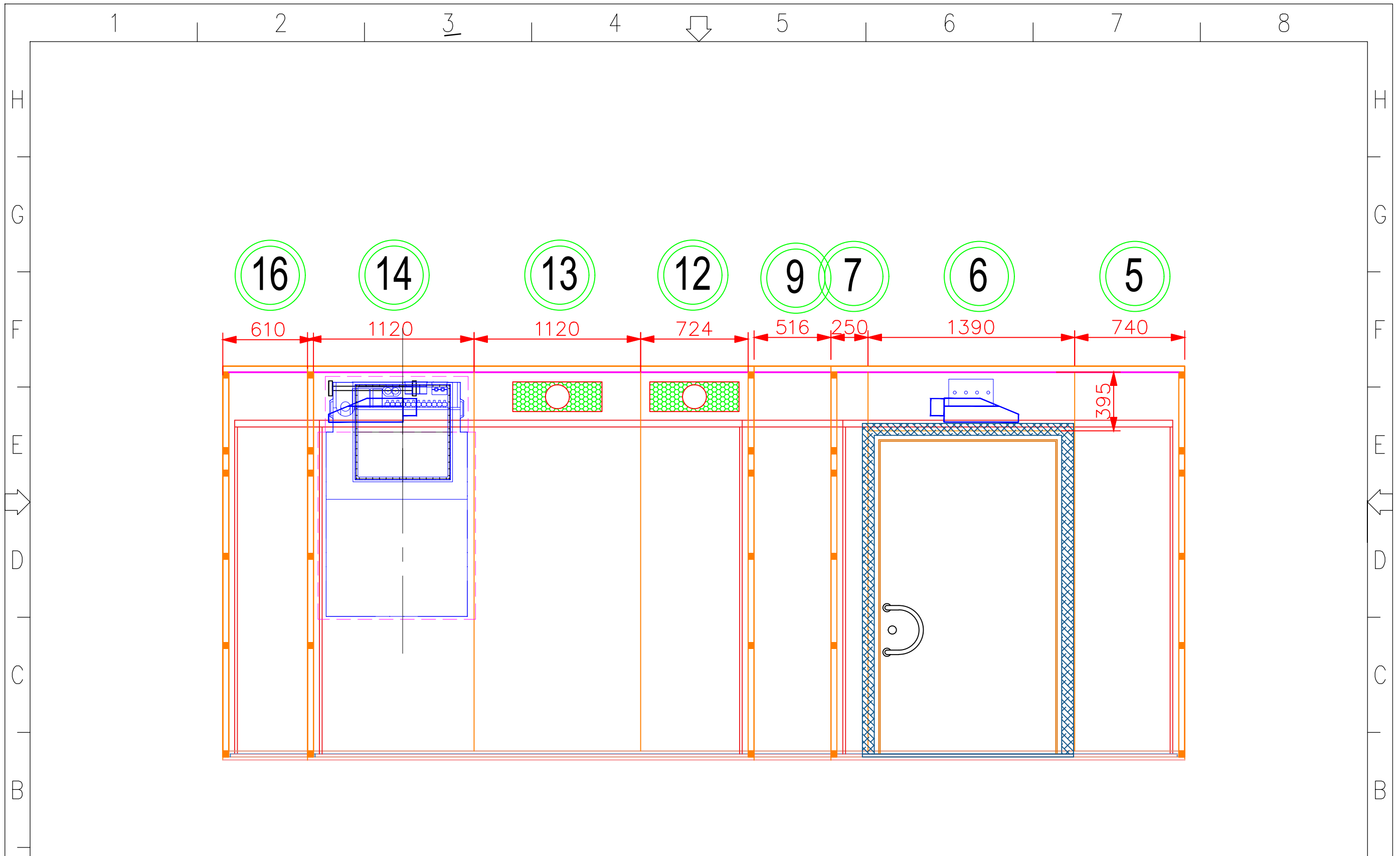

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Size
A3

Dwg No: -

Dwg By: -

All dimensions are millimeters.

Customer/Project Name
Cleopatra Hospital
Cairo, Egypt


Model Type
Ingenia Ambition S

WALL PANELS LEFT VIEW-B



A

A

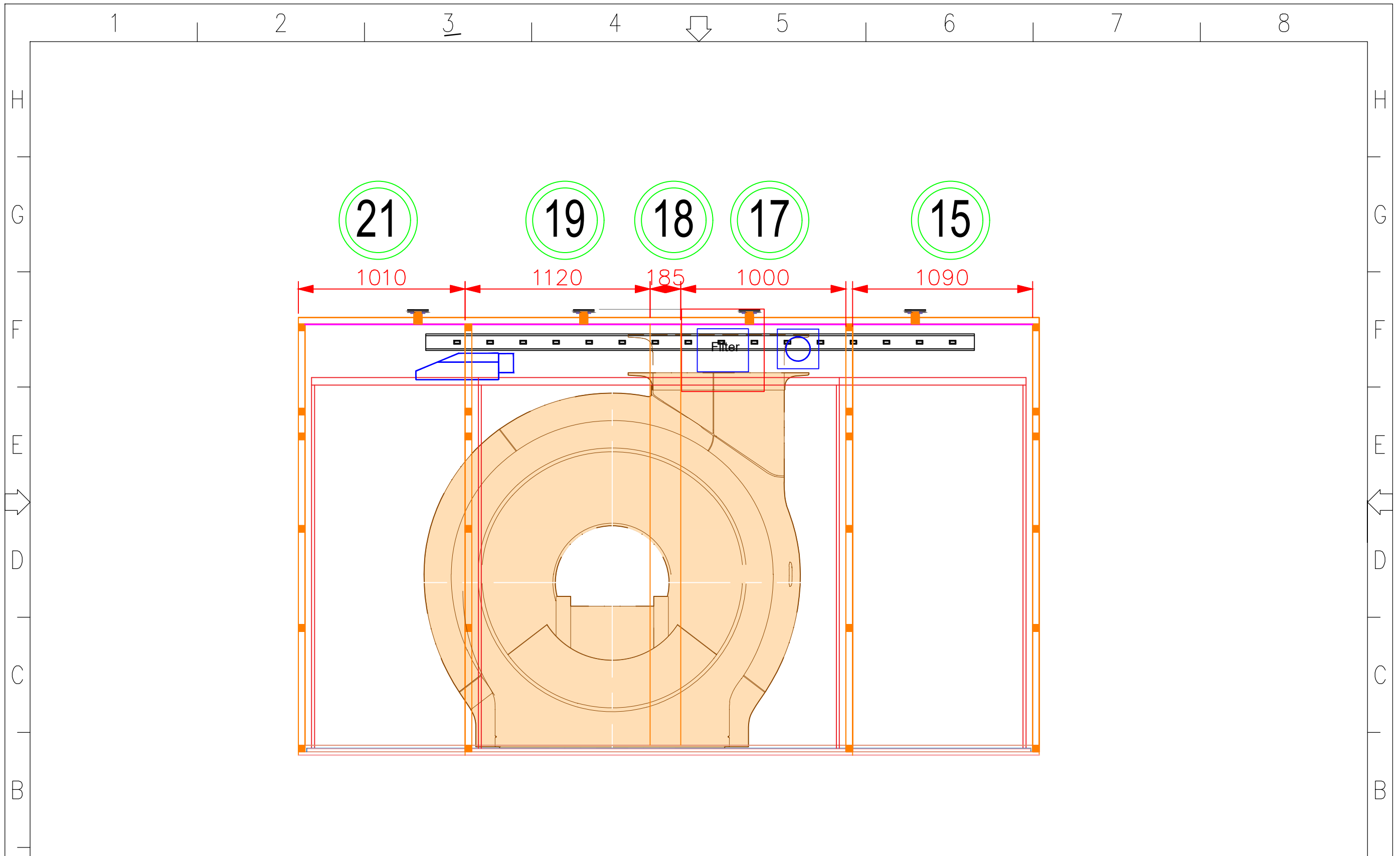

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Dwg No: -

Dwg By: -

All dimensions are millimeters.

Customer/Project Name
Cleopatra Hospital
Cairo, Egypt

Model Type
Ingenia Ambition S


WALL PANELS BEHIND VIEW-C

Page

16



Project No: 2020xxx		Dwg No: -		Customer/Project Name Cleopatra Hospital Cairo, Egypt		WALL PANELS BEHIND VIEW-C		Page 16		ceren CONTRACTING & SERVICES	
Rev No: 2		Dwg By: -		Model Type Ingenia Ambition S							
Date: 02.11.2022		All dimensions are millimeters.									
Scale 1/50											
Size A3											

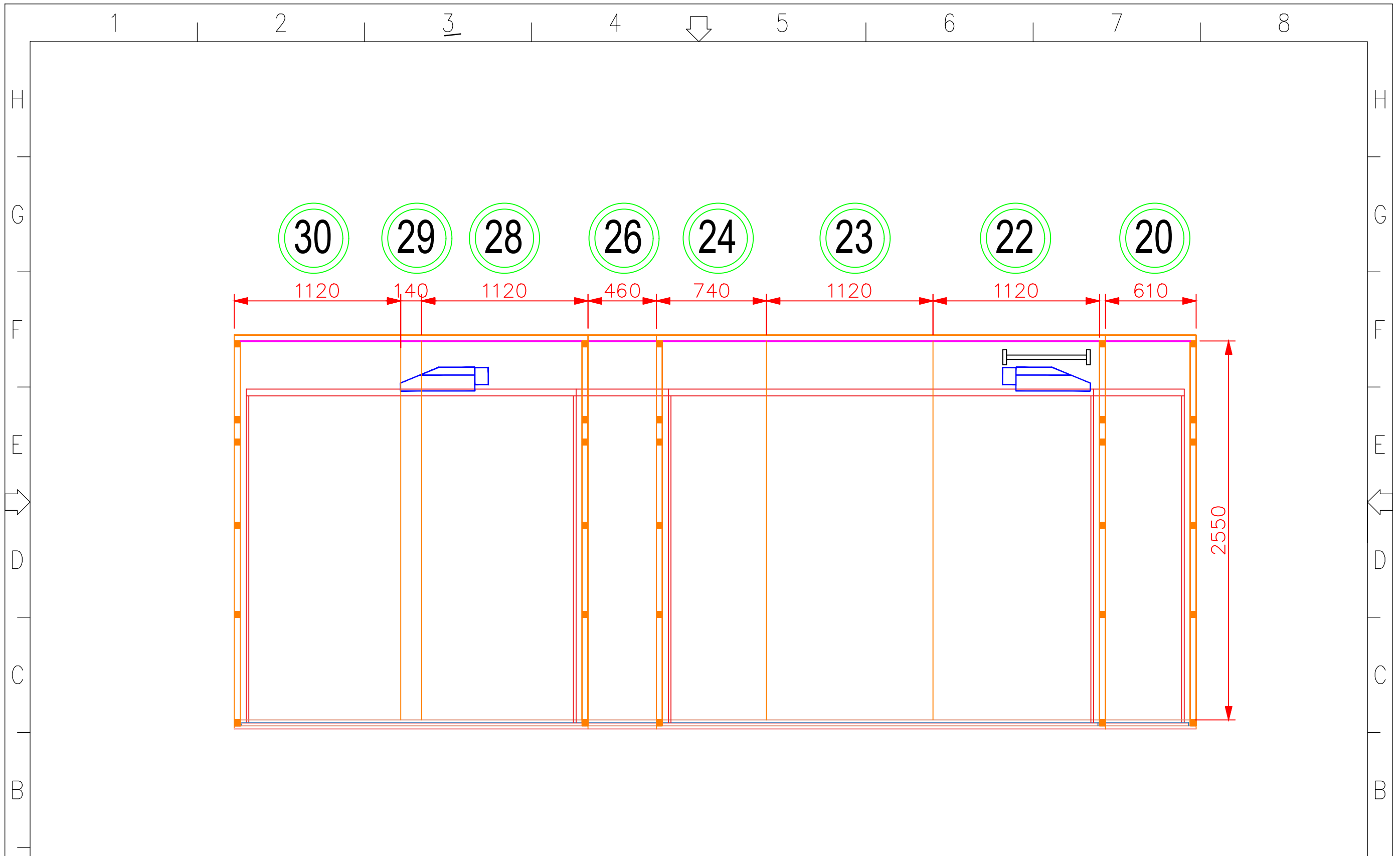

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Dwg No: -

Dwg By: -

All dimensions are millimeters.



Customer/Project Name
Cleopatra Hospital
Cairo, Egypt

Model Type
Ingenia Ambition S


WALL PANELS RIGHT VIEW-D




Project No: 2020xxx		Dwg No: -		Customer/Project Name Cleopatra Hospital Cairo, Egypt		Page 17			
Rev No: 2		Dwg By: -		Model Type Ingenia Ambition S					
Date: 02.11.2022		All dimensions are millimeters.							
Scale 1/50	Size A3								

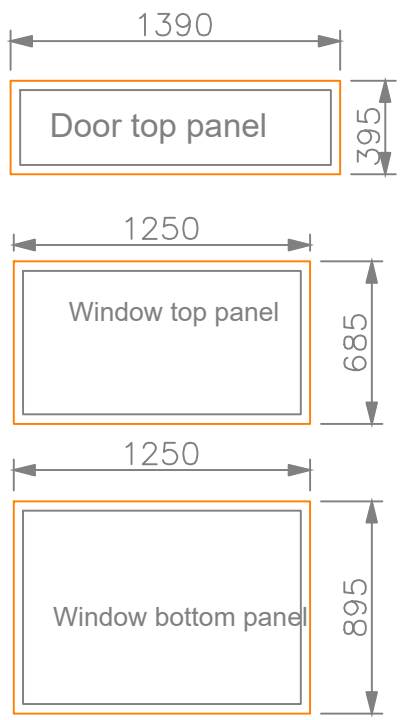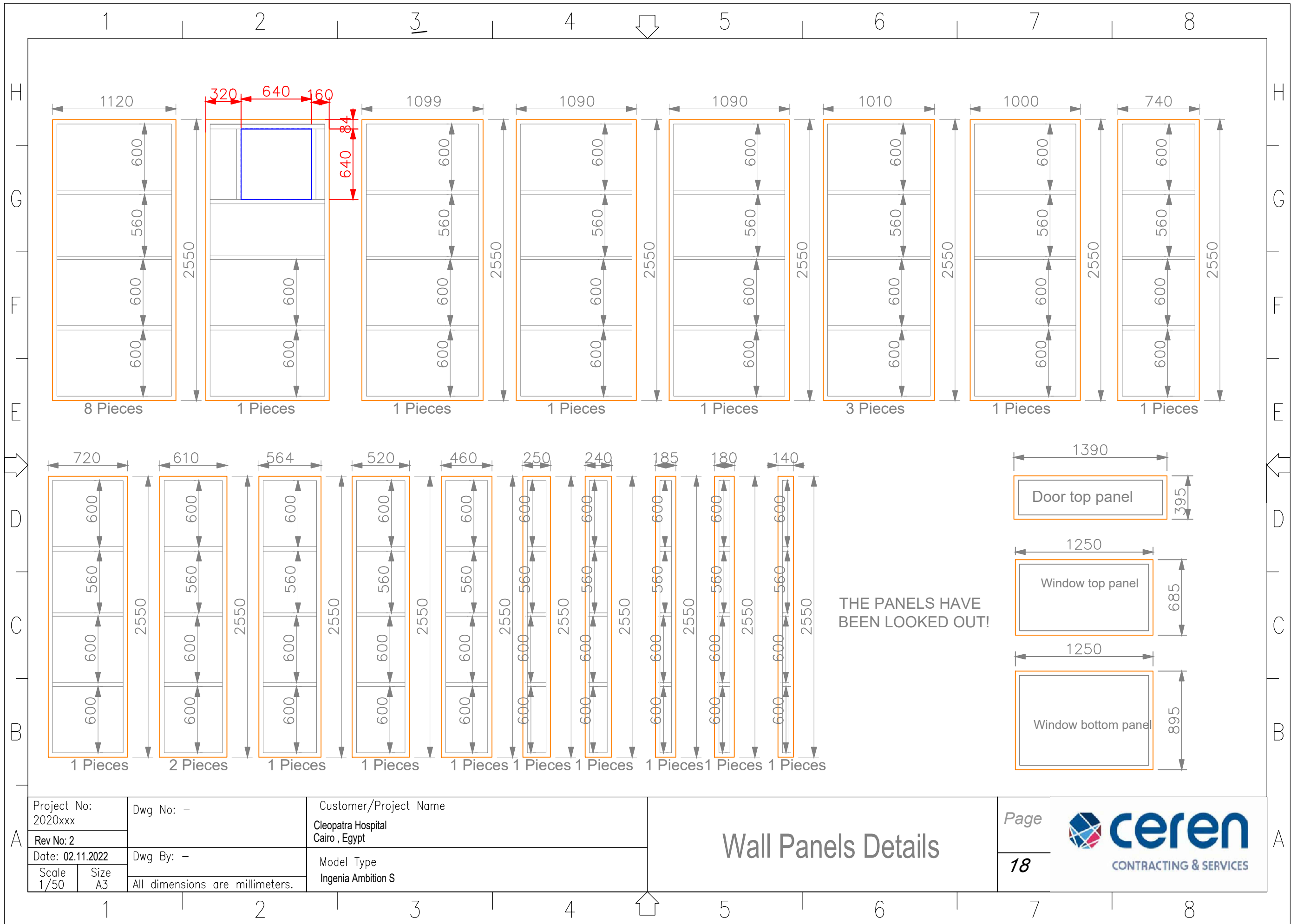

THE PANELS HAVE BEEN LOOKED OUT!

Project No: 2020xxx	Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	<h1>Wall Panels Details</h1>	Page	 ceren CONTRACTING & SERVICES
Rev No: 2	Dwg By: -	Model Type Ingenia Ambition S		18	
Date: 02.11.2022	All dimensions are millimeters.				
Scale 1/50	Size A3				

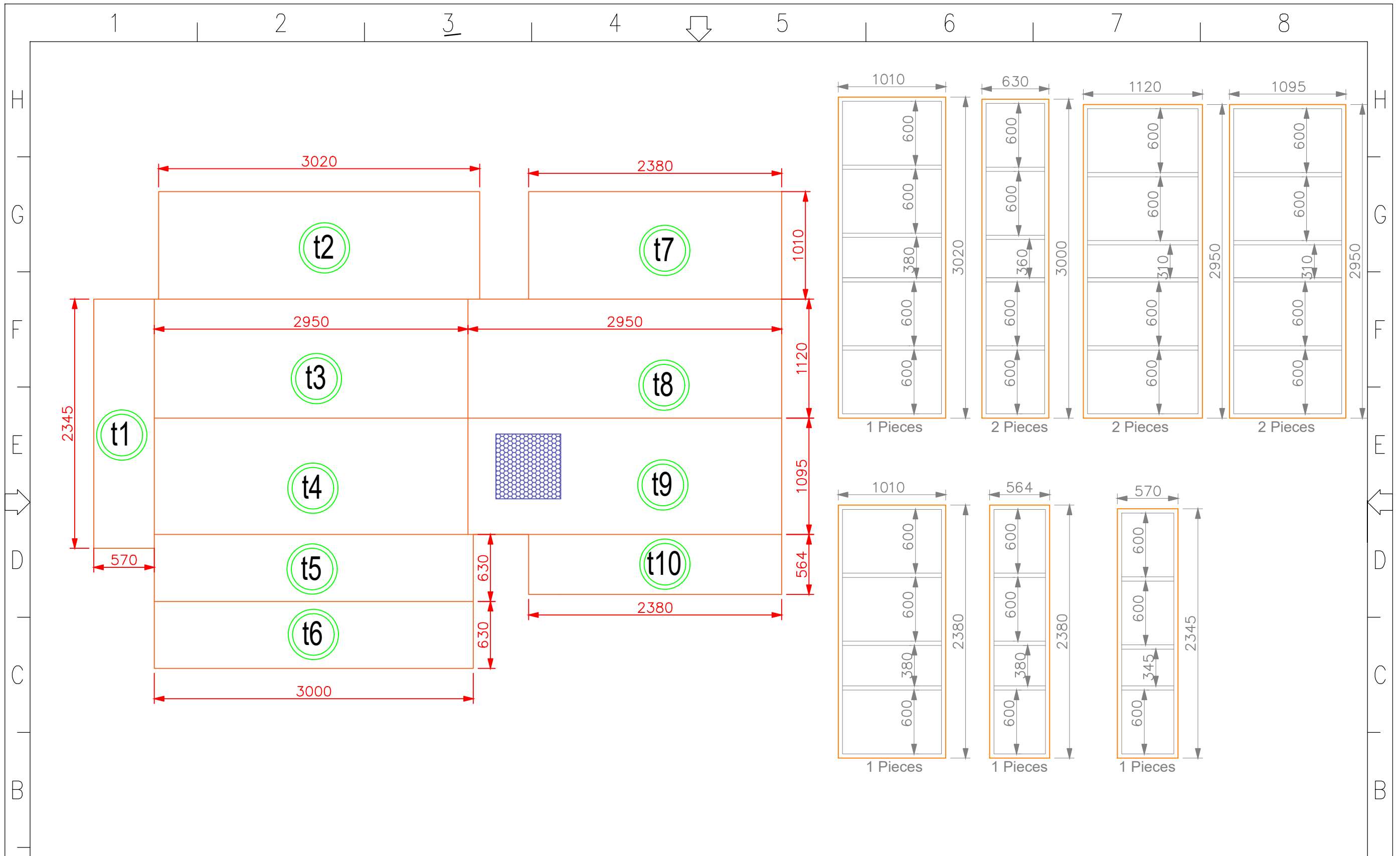

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Dwg No: -

Dwg By: -

All dimensions are millimeters.

Customer/Project Name
Cleopatra Hospital
Cairo , Egypt

Model Type
Ingenia Ambition S

CEILING PANELS DETAILS


Page


19




A

A

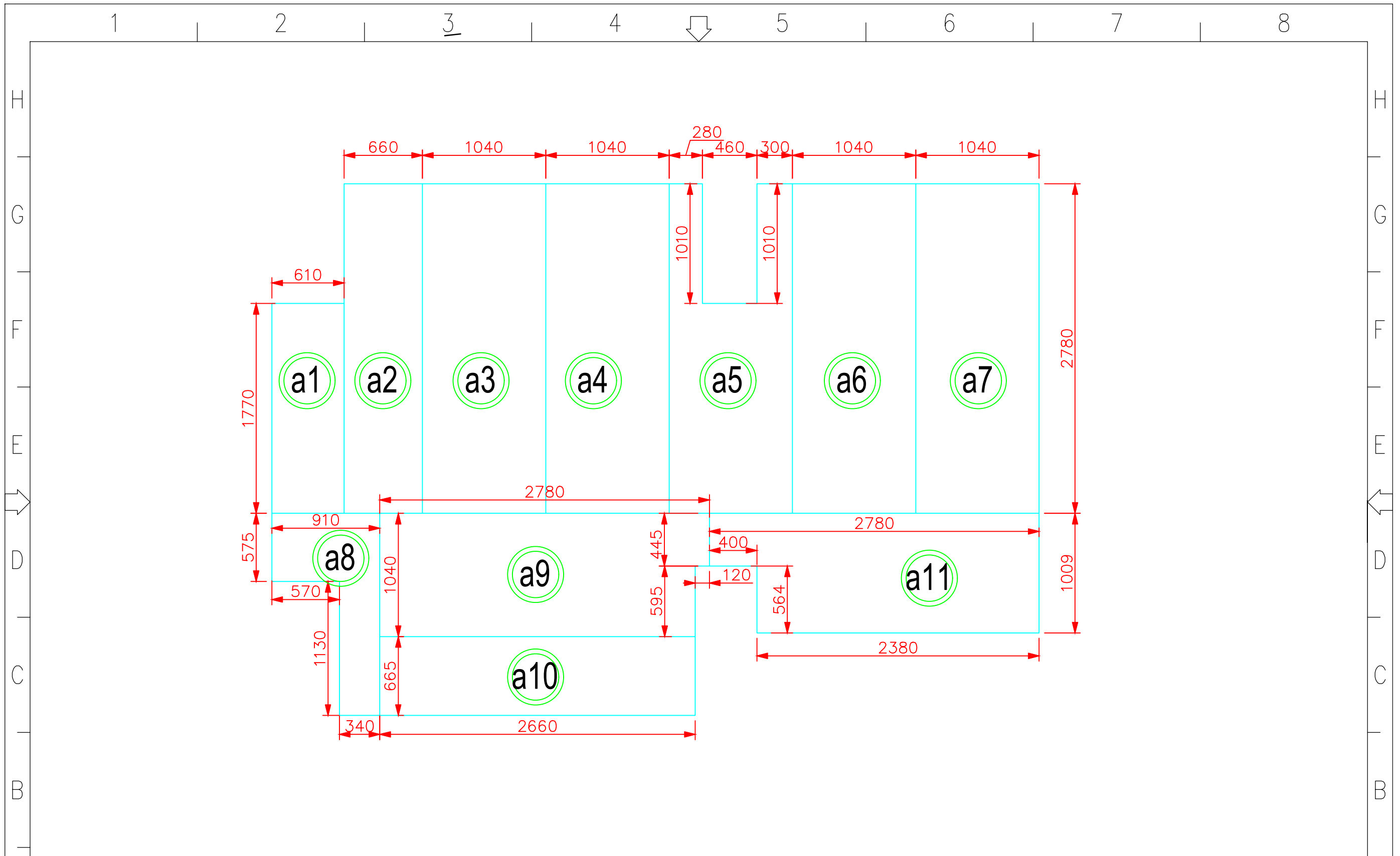

Project No: 2020xxx	Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo, Egypt	Lower floor ground 18mm MDF	Page	 CONTRACTING & SERVICES
Rev No: 2				20	
Date: 02.11.2022	Dwg By: -	Model Type Ingenia Ambition S			
Scale 1/50	Size A3	All dimensions are millimeters.			



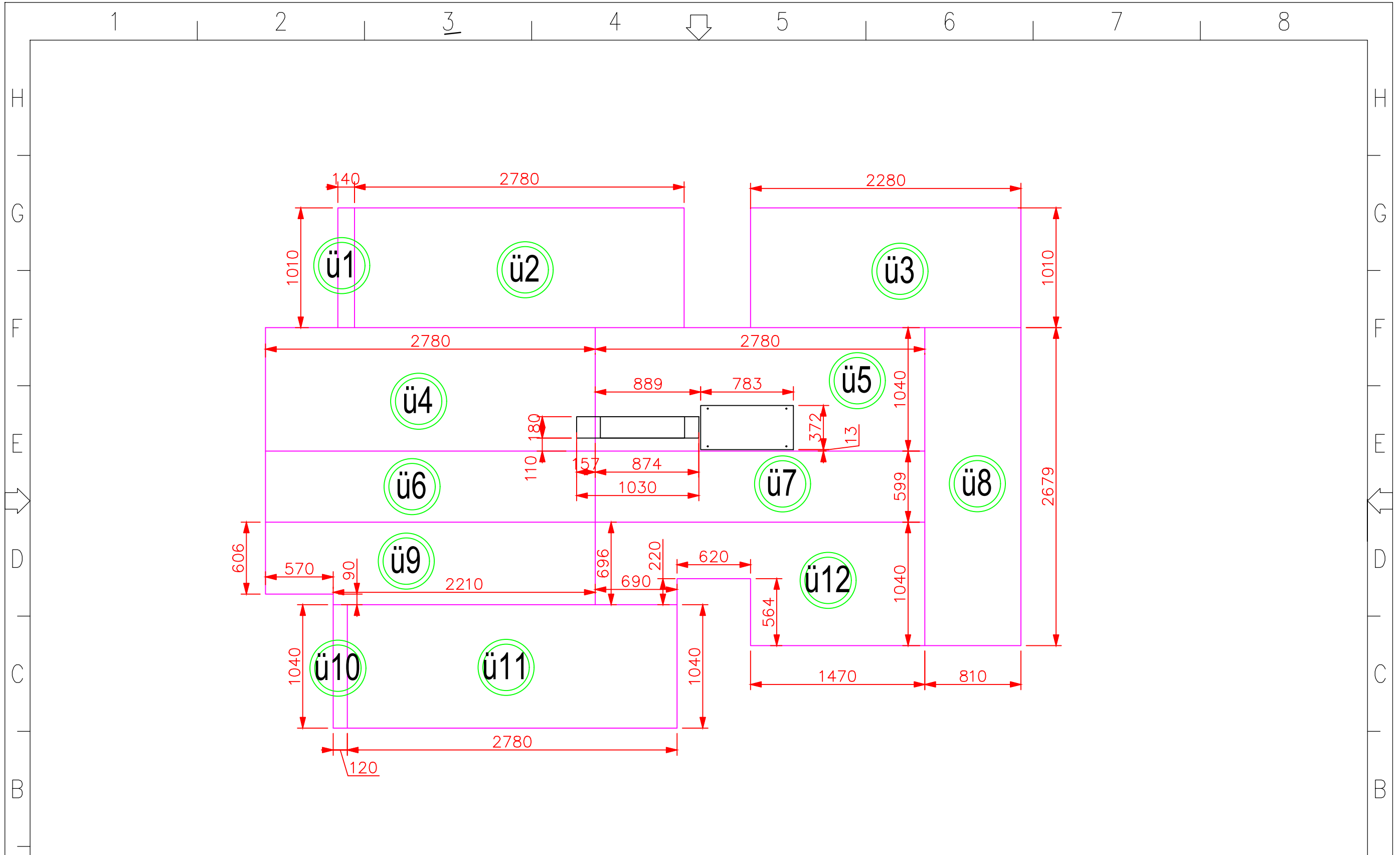

Project No:
2020xxx
Rev No: 2
Date: 02.11.2022
Scale 1/50
Size A3

Dwg No: -
Dwg By: -
All dimensions are millimeters.


Customer/Project Name
Cleopatra Hospital
Cairo , Egypt
Model Type
Ingenia Ambition S

Upper floor ground 18mm MDF

Page
21



Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo , Egypt		Upper floor ground 18mm MDF	Page	
Rev No: 2			Model Type Ingenia Ambition S			21	
Date: 02.11.2022		Dwg By: -					
Scale 1/50	Size A3	All dimensions are millimeters.					

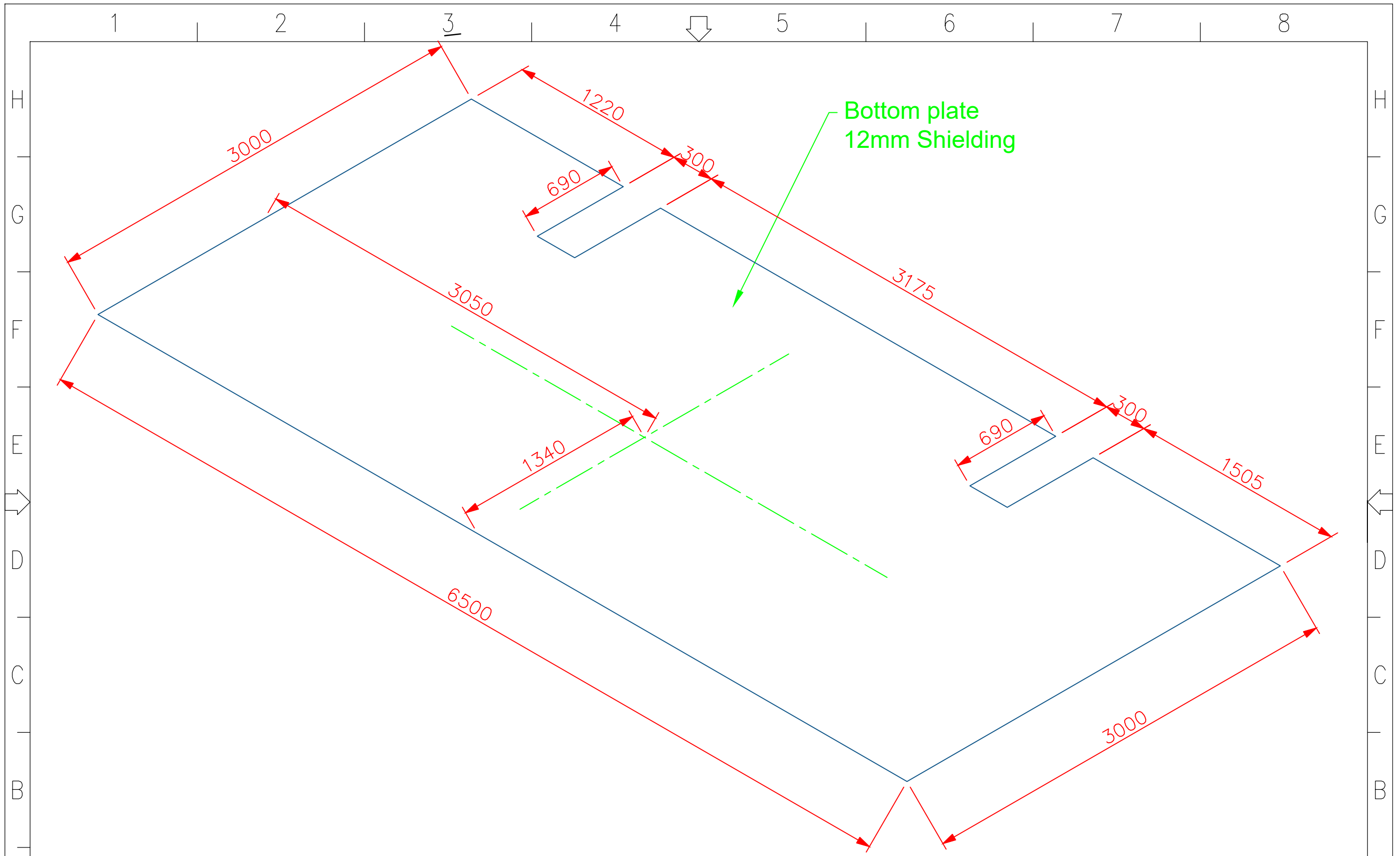

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Dwg No: -

Dwg By: -

All dimensions are millimeters.


Customer/Project Name
Cleopatra Hospital
Cairo , Egypt

Model Type
Ingenia Ambition S

Magnet Shielding

Page
22



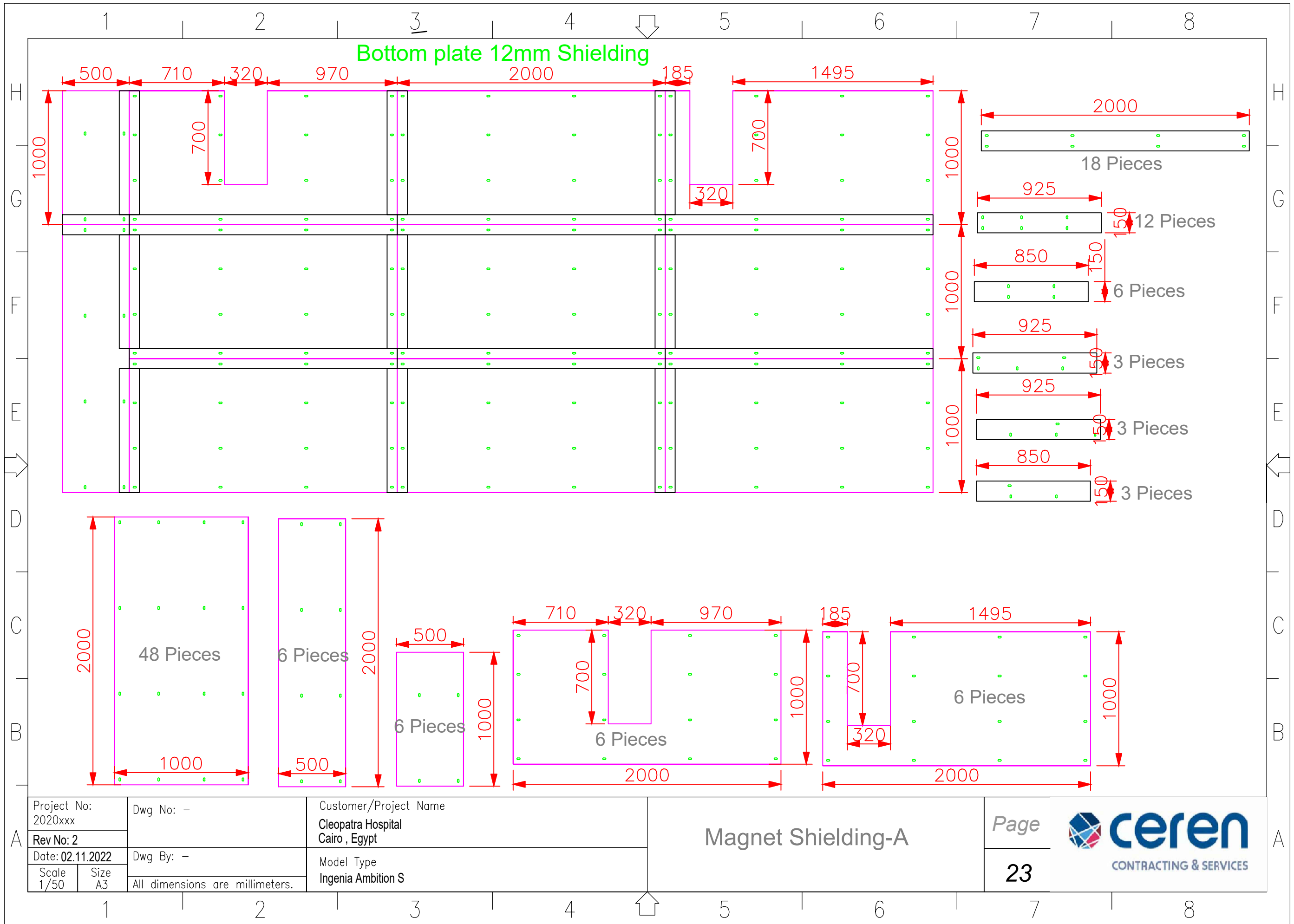

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale: 1/50

Dwg No: -

Dwg By: -


All dimensions are millimeters.

Customer/Project Name
Cleopatra Hospital
Cairo , Egypt

Model Type
Ingenia Ambition S


Magnet Shielding-A




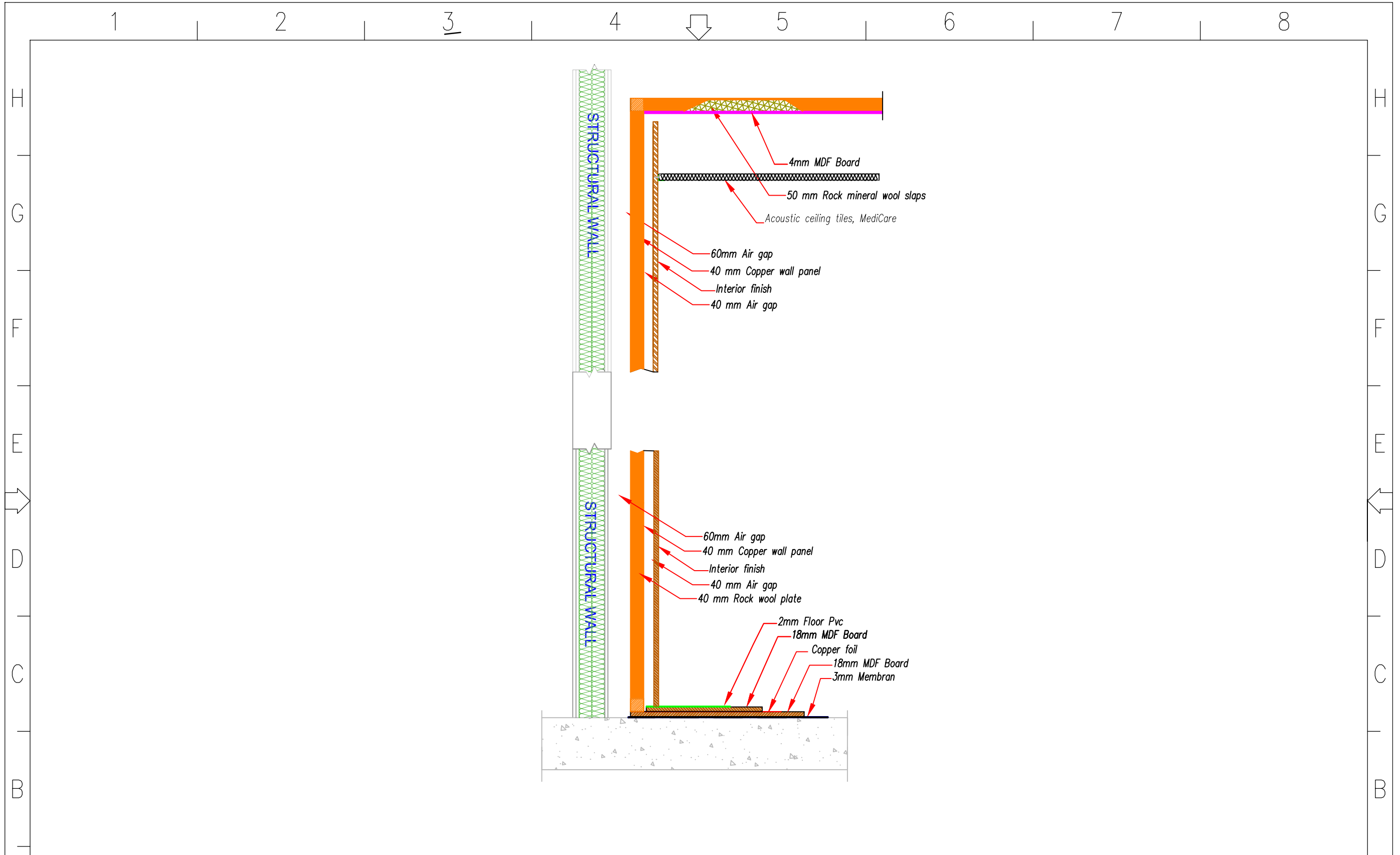

Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo , Egypt
Rev No: 2			
Date: 02.11.2022		Dwg By: -	Model Type Ingenia Ambition S
Scale 1/50	Size A3		

RF Enclosure Details, Wall Panel

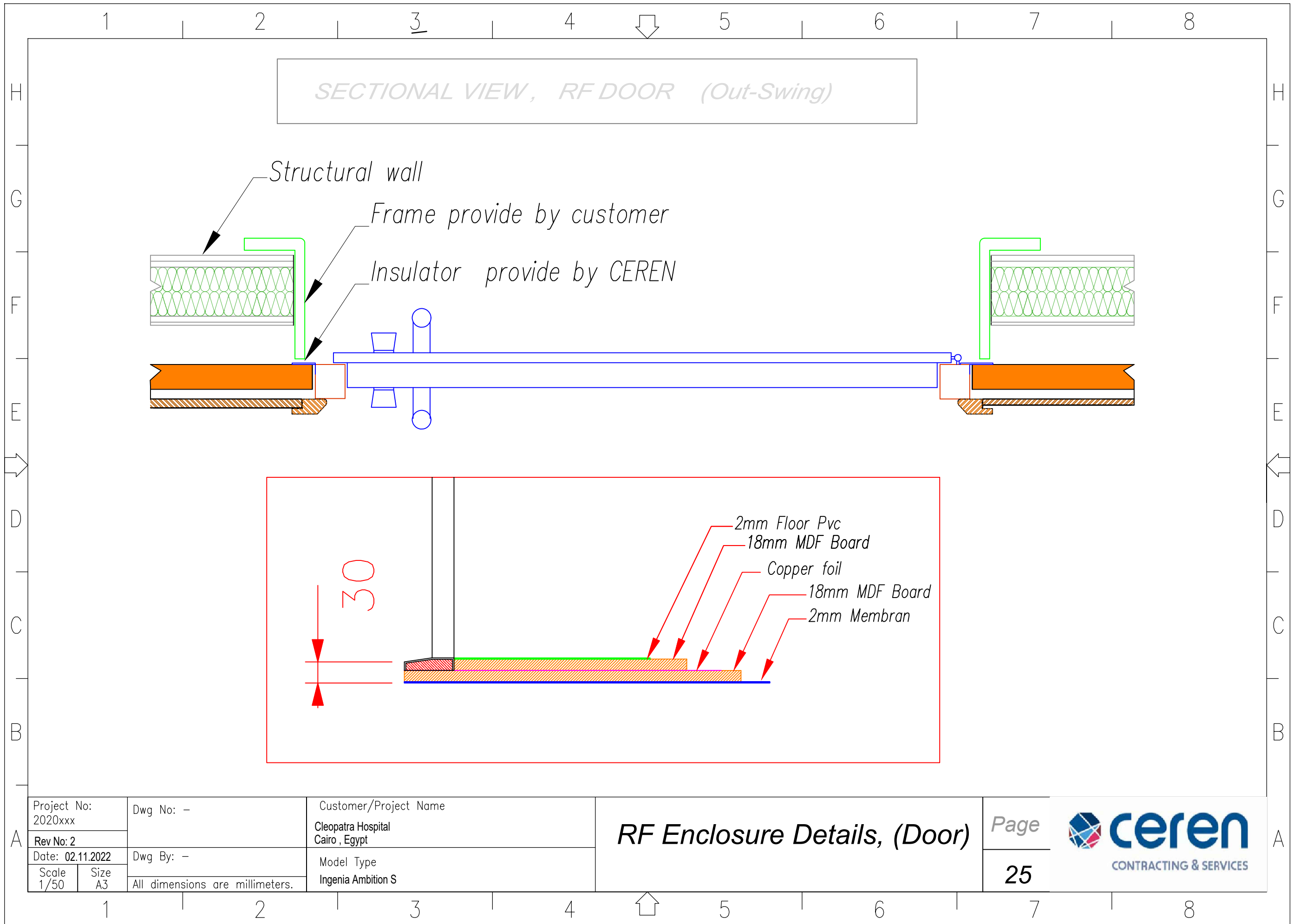

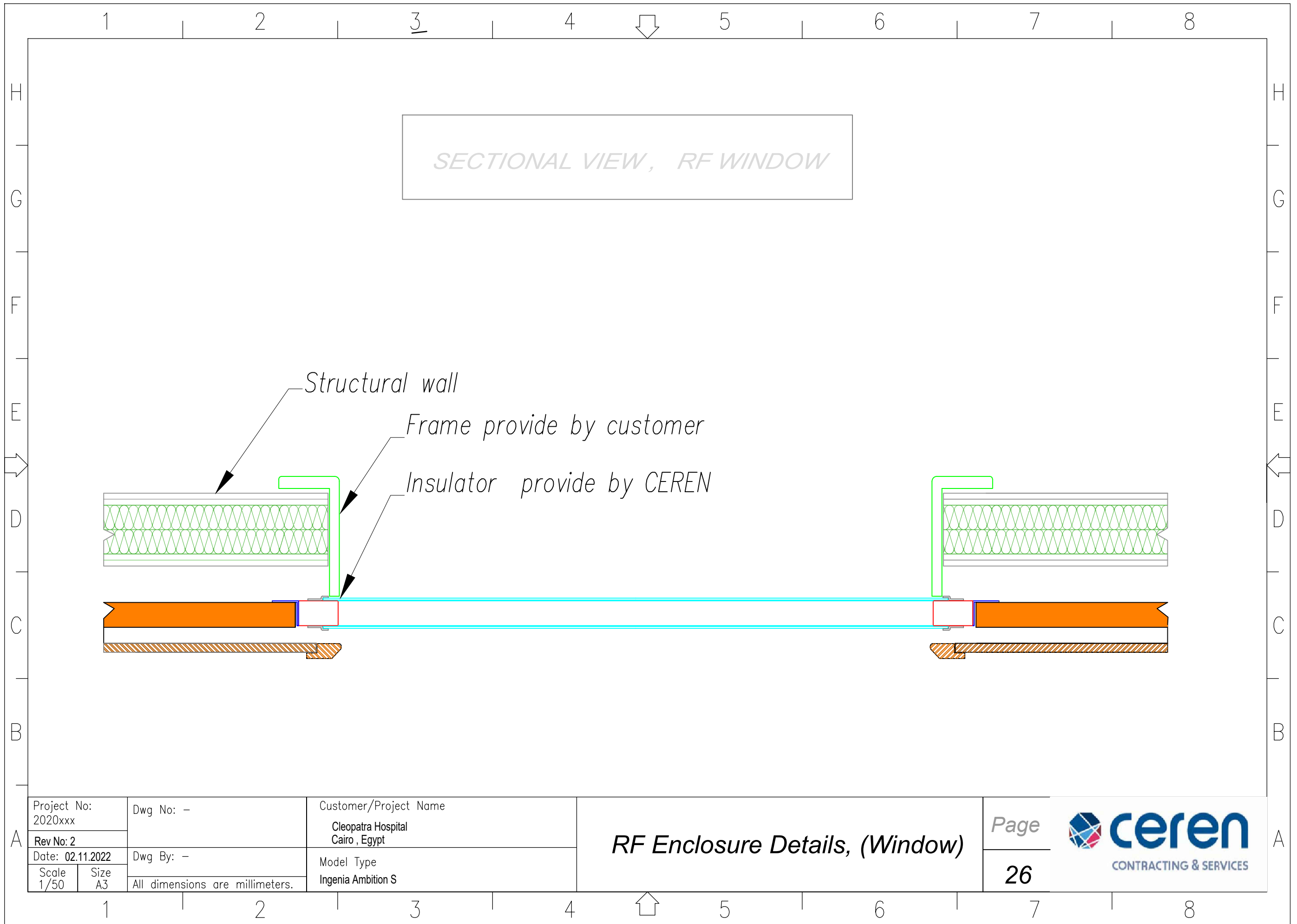

Project No:
2020xxx

Rev No: 2

Date: 02.11.2022

Scale
1/50

Size
A3

Dwg No: -

Dwg By: -

All dimensions are millimeters.

Customer/Project Name
Cleopatra Hospital
Cairo, Egypt

Model Type
Ingenia Ambition S

RF Enclosure Details, (Window)




A

1 2 3 4 5 6 7 8


H H

G G



Top view

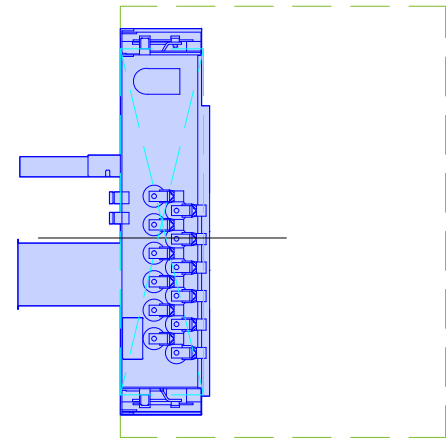

Fronte view

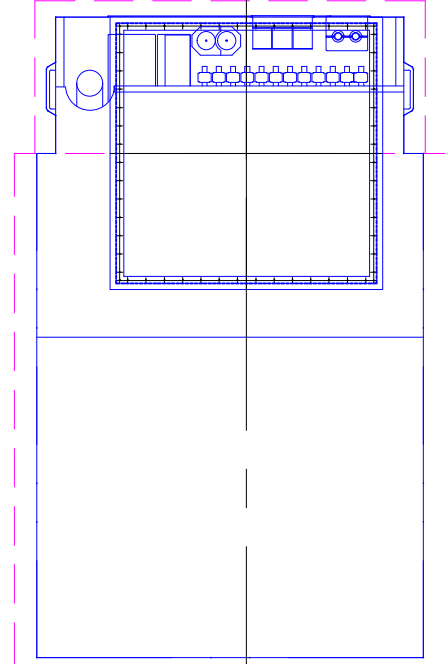

Side view

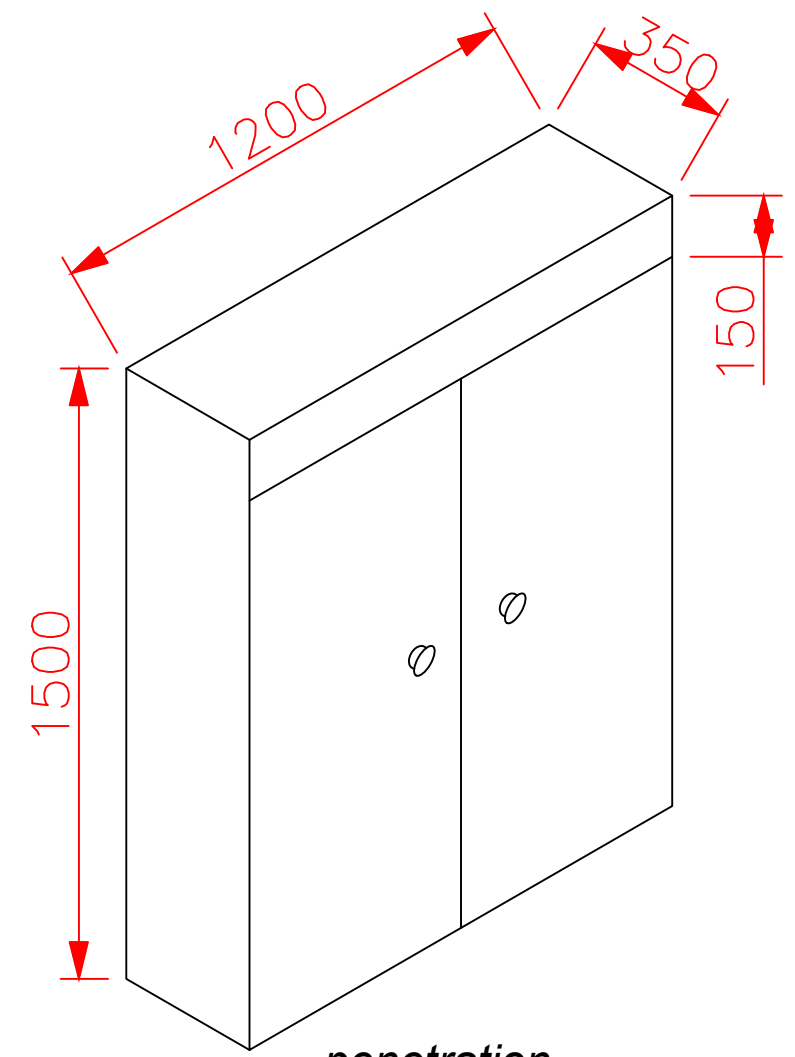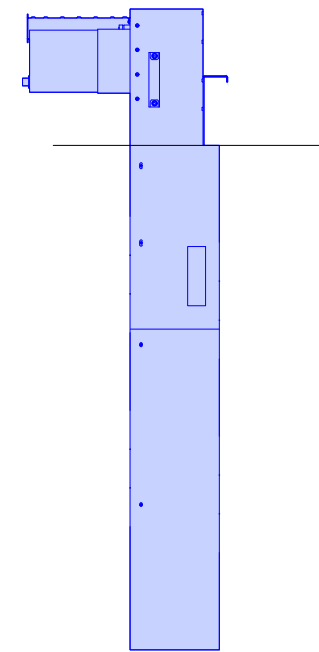

penetration locker!

D D

C C

B B


Project No: 2020xxx		Dwg No: -	Customer/Project Name Cleopatra Hospital Cairo , Egypt		Page 27
Rev No: 2		Dwg By: -	Model Type Ingenia Ambition S		
Date: 02.11.2022	All dimensions are millimeters.				
Scale 1/50	Size A3				



1 2 3 4 5 6 7 8

A A

H H

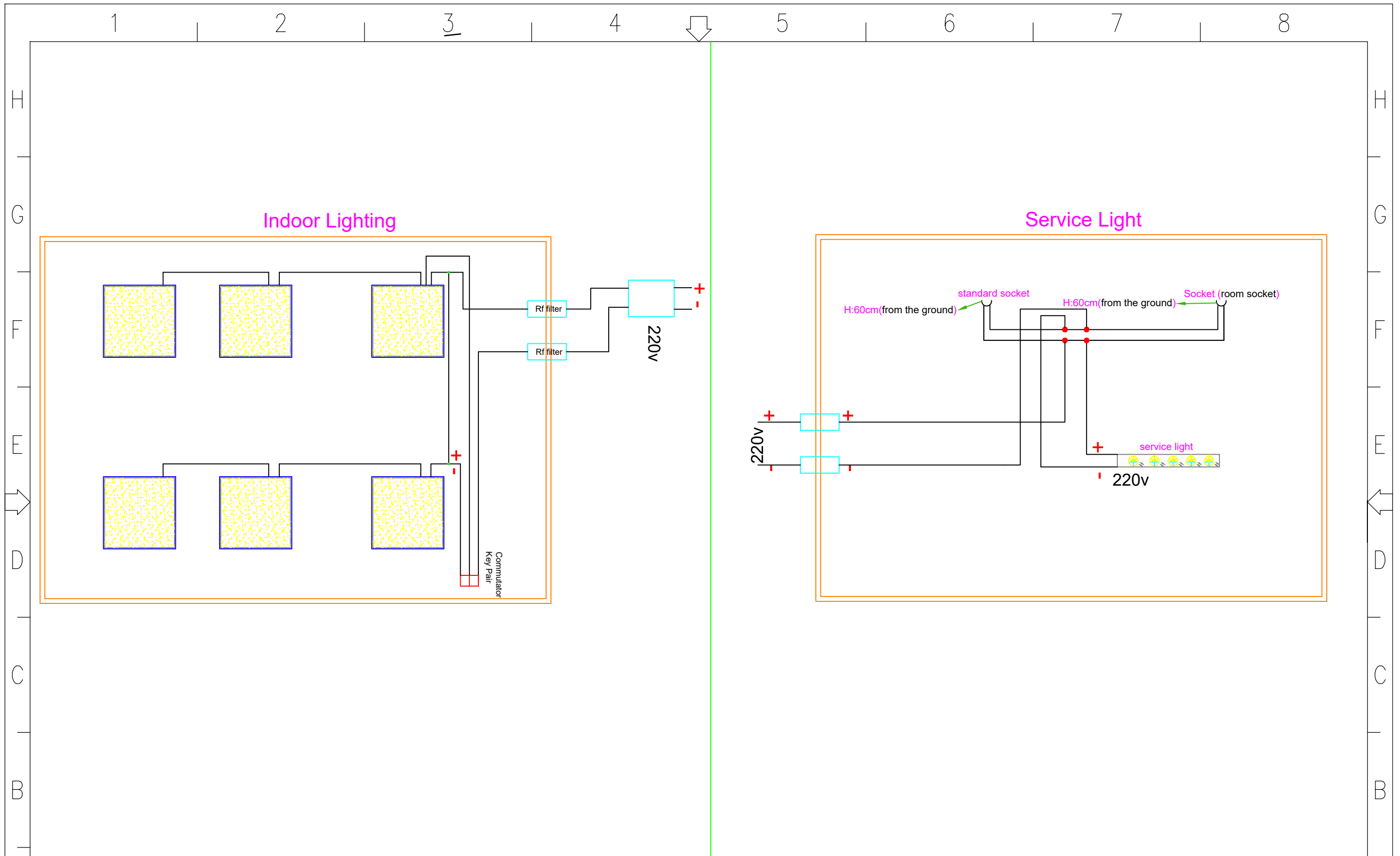

Project No: 2020xxx	
Rev No: 2	
Date: 02.11.2022	
Scale 1/50	Size A3

Dwg No: -	Dwg By: -
All dimensions are millimeters.	

Customer/Project Name Cleopatra Hospital Cairo , Egypt	Model Type Ingenia Ambition S
--	----------------------------------

Room lighting drawing

Page
28



A